

Janis Care Private Limited, inception in May 2021, Janis a specialised manufacturing company specialising in the Manufacture and Marketing of Surgical and Non-Surgical products having our Head office in Mumbai, and Regional Sales office in New Delhi, Pune, Aurangabad and Jaipur. A total team of 45 highly enthusiastic individuals thriving to achieve the highest standard of Quality and Service to our customers.

We proudly state that our production process and manufacturing set up is as per International Standards and we take optimum care in maintaining finest Hygiene and Quality Standards towards our products. From our manufacturing, packaging to logistics we have put our thoughts on every detail and planned the process with our customers in mind.

Janis Care as a brand is here to stay and make a dent in the surgical and medicine industry. We strive to become the most preferred choice of brand for all our consumers, not because we marketed it well, but because they got more than what they expected. That's Janis Care.

From CEO Desk



Mr. Pavitra Anand

JANIS is headed by Mr. Pavitra Anand who has been an Investment banker and has been involved in the Indian Capital Markets for the past 24 years. Mr. Anand has served and gained experience and in depth knowledge in several assignments roles working in Multinational banks like Citibank N.A., ABN Amro NV, Societe Generale and Barclays.

Anand has gained experience in the field of Credit Risk - Investment/Corporate Banking, Equity Portfolio Management, setting up Structured Product Platform, and Wealth Management/Financial Management. Anand has also gained In-depth knowledge on Debt Rating Model, to evaluate the credit worthiness of large, medium and small corporate. Putting forth 24 years of experience Mr. Anand has been instrumental in turning around sick companies in the field of aviation, healthcare etc. Apart from startegy ties up, Mr. Anand has been involved in Fund Raising in the form of debt and Equity for SME and MME in the field of Pharmaceuticals, Real Estate, BPO (Business Process Units), Chemical Industry, Hospitality and Education.

Mr. Anand has earned a double Master Degree in Business Administration specialising in Innovation and Entrepreneurship and Masters in Applied Finance specialising in Portfolio Management and Corporate Finance from Melbourne, Australia.

CEO

Pavitra Anand

Janis Care



01. Dressing & Wound Management Products

- Absorbent Cotton Rolls 03
- Janis Care Cotton Roll Bandage 04
- Janis Sterile Gauze Swab 05
- Non-Sterile Gauze Swab 06
- Gauze Than 07
- Janis Crepe Bandage B.P 08
- Janis Elastic Adhesive Bandage 09
- Janis Combine Dressing 10
- Janis Care Abdominal MOP 11
- Gamjee Roll 12
- Janis Microporous Paper Tape 13
- Janisocare POP 14
- Janis I.V. Cannula Fixator 15
- Transparent Dressing 16

02. Surgical Disposable Products

- Janis Care Surgical Mask 19
- Janis Care N95 Mask 20
- Janis Care Shoe Cover & Cap 21
- Surgical Latex Examination Gloves 22
- Transducer Protector 23
- Janis Dialysis Blood Tubing Set - Post Pump 24
- I.V. Set (Infusion Set) 25
- Blood Transfusion Set (B.T. Set) 26
- Syringes & Needle 27
- Spinal Needle 28
- I.V. Cannula 29
- Three Way Stop Cock 30
- Plain Sheet 31
- Underpads 32
- Bath Wipes 33
- Cord Clamp 34

03. Antiseptics & Disinfectants Products

- Janisodine Wound Care - 5% & 10% 37
- Janisodine Wound Care Scrub - 7.5% 38
- Clens Antiseptics Solution 39
- Clens Care Plus 40
- Janzyme 41
- Janis Shield (Instant Hand Disinfectant) 42
- Rust Remover 43
- Hemodialysis Concentrate Solution (Part A & Part B) 44

04. Medical Device

- Janis U Safe Forte 47
- Jancitro 21% 48
- Jancitric 50% 49
- Janis Kidney Plus 50
- Janis Guard (Surface Disinfectant) 51
- Nebulizer - Mini / Prime 52
- Digital Thermometer 53
- Steamer 54
- Hot Water Bottles 55
- Glucometer With Strip 56
- Blood Pressure Machine Electric Chargeable / Cells 57

05. Personal Care

- Adult Diaper 60
- Adult Diaper - Pants / Pull-Up 61
- Sanitary Pads - Regular 62
- Maternity Pads 63

06.

Anesthesia, Respiratory Care & Miscellaneous Products

● Janis Neso Mist – (Nebulizer Accessories Kit)	66
● Teflon Nebulizer Mask (Child,Adult)	67
● Janis Oxee Mask – (Flexible Oxygen Face Mask)	68
● Teflon Oxygen Mask (Child,Adult)	69
● Janis Oxee Set – (Nasal Oxygen Cannula)	70
● HI Concentration Oxy Mask (HI Mask)	71
● Venturi Mask With Adjustment Diluter	72
● Pressure Extension Line – PMO Line Pressure	73
● Extension Line – Vein O Line	74
● Abdominal Drainage Kit (ADK)	75
● Janis Suction Set (Yankauer Suction Set)	76
● Reusable Ambu Bags	77
● Three Balls Spirometer	78
● Nibp Cuff	79
● Disposal Monopolar Pencils 3 – pin	80
● Infant Feeding Tube – Feedy	81
● Ryles Tube	82
● Endotracheal Tube Plain	83
● Endotracheal Tube Cuffed	84
● Oro – Pharyngeal Airway (Guedel Airway)	85
● Suction Catheter Plain	86
● Foley Catheter – 2 Way	87
● Silicon Foley Catheter – 2 Way	98
● Urine Collection Bag – Uro Bag	89
● Urine Bag With Measured Volume Chamber	90

● Universal Shoulder Mobilizer	99
● Wrist Wrap With Thumb	100
● Ankle Support	101
● Hand Stick Support – Single & 4 Legs	102

08

Parenteral Drugs (Injections)

INJECTIONS

● Janitropine (Atropine INJ IP)	105
● Janikacin (Amikacin Sulphate INJ 250 mg)	105
● Janikacin (Amikacin Sulphate INJ 500 mg)	105
● Janicetam (Piracetam INJ)	106
● Janthamin (Thiamine INJ IP)	106
● Janistidine (Ranitidine INJ)	106
● Janifenac (Diclofenac Sodium INJ IP)	107
● Jantamicin (Gentamicin Sulphate INJ)	107
● Janifer (Iron Sucrose INJ 20 mg)	107
● Janisparin (Heparin 5000 IU / 5 ml)	108
● Janisparin (Heparin 25000 IU / 5 ml)	108
● Janispan (Pantaprazole INJ)	108
● Janispenem (Meropenem INJ 1 g)	109
● Janispenem (Meropenem INJ 500 mg)	109
● Jantriaxone (Ceftriaxone INJ 1 g)	109
● Janicycline (Doxycycline INJ USP)	110
● Janitaz (Piperacillin & Tazobactam INJ 4.5 g)	110
● Janitrone (Ondansetron INJ IP)	110

Fluids

● J D5 – L – 500 ml	111
● Janis Metzole IV 500 mg	111
● Janispar IV 100 ml	111
● J NS – L – 100 ml, 500 ml & 1000 ml	112
● J DNS – L – 500 ml	112
● Janisrin L – 500 ml	112

07.

Orthopedic Products

● Abdominal Belt	93
● Arm Sling	94
● Cervical Collar With Neck Support	95
● Knee Cap	96
● Lumbo Sacral Belt	97
● Rib Belt	98

09. Baby Care

- Baby Diapers 115
- Baby Diapers - Pants / Pull-up 116
- Baby Wipes 117

10. Medicines

ETHICAL MEDICINES

- ~ Analgesics / Antipyretics
- Paigo 500 mg - Paracetamol 500 mg 120
- Paigo 650 mg - Paracetamol 650 mg 120
- Paigo Plus - Aceclofenac 100 mg + Paracetamol 325 mg 120
- Paigo Plus SP - Aceclofenac 100 mg + Paracetamol 325 mg + Serratiopeptidase 15 mg 121

~ Anti Reflux

- Papol D - Pantoprazole 40 mg + Domperidone 30mg 121

~ Erectile Dysfunction

- Jumpon - L 50 mg - Sildenafil Citrate (Oxide Blue) 50 mg 122
- Jumpon - S 100 mg - Sildenafil Citrate (Oxide Blue) 100 mg 122
- Erectod 20 mg - Tadalafil 20 mg 122

GENERIC MEDICINES

- ~ Calcium Supplement
- Janical - D3 - Calcium & Vitamin D3 123
- Janistite Syrup - Cyproheptadine hydrochloride 2 mg + Tricholine Citrate 0.275 gm + Sorbital Solution 2 gm 123

~ Anti Reflux

- Janisec - D - Pantoprazole 40 mg + Domperidone 10 mg 123

~ Analgesics / Antipyretics

- Janidol 500 mg - Paracetamol 500 mg 124
- Janidol 650 mg - Paracetamol 650 mg 124
- Janidol Plus - Aceclofenac 100 mg + Paracetamol 325 mg 124
- Janidol Plus SP - Aceclofenac 100 mg + Paracetamol 325 mg + Serratiopeptidase 15 mg 125
- Janifenac - P - Diclofenac 50 mg + Paracetamol 325 mg 125
- Janispas - Mefenamic Acid Tablet 500 mg 125

~ Analgesic / Anti Cold Products

- Janimont - L - Levocetirizine Hcl 5 mg + Montelukast Sodium 10 mg 126
- Janicet - Cetirizine Dihydrochloride 10 mg 126
- Janispas - D - Dicyclomine Hydrochloride 10 mg + Mefenamic Acid 250 mg 126

~ Anti Fungal Infection & Anti Bacterial Infection

- Janiclo Dusting Powder - Clotrimazole (1% w/w) 127
- Janiclo 1% Cream - Clotrimazole (1% w/w) 127
- Janiclo - B Cream - Beclomethasone (0.025 % w/w) + Clotrimazole (1% w/w) 127

~ Erectile Dysfunction

- Janirec 20 mg - Tadalafil 20 mg 128
- Janion - L 50 mg - Sildenafil Citrate (Oxide Blue) 50 mg 128
- Janion - S 100 mg - Sildenafil Citrate (Oxide Blue) 100 mg 128

~ Antibiotics

- Janithro 250 mg - Azithromycin 250 mg 129
- Janithro 500 mg - Azithromycin 500 mg 129
- Janifloxin - Ofloxacin 200 mg + Ornindazole 500 mg 129

11. Others

- Dialyzer 132
- AV Fistula Needle 133
- Sodium Hypochlorite 134



Dressing & Wound Management Products



Sizes	Pack Size	MRP
500 gm	1x1	₹ 575
400 gm	1x1	₹ 480
300 gm	1x1	₹ 350
200 gm	1x1	₹ 300
100 gm	1x 6	₹ 175
70 gm	1x 6	₹ 120
50 gm	1x 10	₹ 75
20 gm	1x 20	₹ 30

THE KEY ATTRIBUTES OF ABSORBENT COTTON ROLL ARE AS FOLLOWS:

- **Characteristics/Features**
 - Made from superior short 100% cotton fiber with uniform layers and no uneven edges or loose fibers and texture that promotes absorption of blood and body fluids and hence perfectly suitable as a dressing.
 - Material for wounds and incisions.
- **Uses / Instructions:**
 - Use as required or as directed by the physician.
 - Read the label carefully before use.
 - Store in a cool dry place away from direct sunlight.
 - Keep out of reach of the children.
 - Avoid physical damage.
- **Common Material Used:**
 - Material – 100% pure cotton (*Gossypium spp.*), free from synthetic fibers.
 - Color – White, natural (no optical brighteners or dyes).
 - Odor – Odorless.
 - Appearance – Soft, fluffy, and uniform.



Sizes	Pack Size	MRP
15 cm x 2 mtrs	1 x 12	₹ 456
10 cm x 2 mtrs	1 x 12	₹ 404
7.5 cm x 2 mtrs	1 x 12	₹ 328
5 cm x 2 mtrs	1 x 12	₹ 252

Sizes	Pack Size	MRP
15 cm x 4 mtrs	1 x 12	₹ 456
10 cm x 4 mtrs	1 x 12	₹ 404
7.5 cm x 4 mtrs	1 x 12	₹ 328
5 cm x 4 mtrs	1 x 12	₹ 252

THE KEY ATTRIBUTES OF COTTON ROLL BANDAGE ARE AS FOLLOWS:

- Characteristics/Features**
 - Janis Care cotton roll bandages are made from a thin, woven fabric material that is placed over a wound to keep it clean while it promotes wound healing.
 - It can be used to secure a dressing in place or it can be used directly on a wound.
 - These bandages are available in many sizes. Ideal for covering up difficult to dress areas.
 - When using a roll, the end is pulled out and placed over a piece of wound dressing. Bandage is then wrapped around the location of the wound.
 - Janis Care bandage can be wrapped & tied tightly when compression is needed to control. Read the label carefully before use.
- Uses/Instructions:**
 - Janis care cotton roll bandages are soft, easy to apply, highly absorbent, breathable, hypoallergenic, non irritating and available in various sizes for matching various wound dressing needs.
 - Janis Care bandages effectively absorb fluids preventing soaking, keeping the wound clean and does not cause irritation.
 - Our bandages are suitable for various wound dressing needs, including first aid, sports injuries and wounds.
 - Soft and Comfortable: Made from high-quality, 100% pure cotton to provide gentle contact with the skin and prevent irritation.
 - Highly Absorbent: Capable of efficiently absorbing exudate, blood or fluids, making it useful for wound dressing and cushioning.
- Common Materials Used:**
 - 100% Pure Cotton Yarn.
 - Weave Type - Plain Weave.
 - TPI (Threads Per Inch) - 30 x 20 (Warp x Weft) – Common Size 47 GSM.
 - Yarn - count 20s to 40s (typically 20s or 30s cotton yarn).

***GST will be applicable on the rates of all the products.**





Sizes	Pack Size	MRP
10 cm x 10 cm x 12 ply	1 x 10	₹ 210
7.5 cm x 7.5 cm x 12 ply	1 x 10	₹ 620
5 cm x 5 cm x 12 ply	1 x 10	₹ 650

THE KEY ATTRIBUTES OF STERILE GAUZE SWAB ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Janis Care Sterile Gauze Swab is a soft, strong sterile durable cotton gauze swab.
 - Provides high level of absorption of body fluids with low adhesion properties.
 - Provided with large lids and is double wrapped with a covering to avoid contamination.
 - Ideal for wound dressing or for absorbing blood and body fluids during a surgery.
 - Useful for blood collection as it provides clean surface to apply pressure to venipuncture site after the needle is removed.
 - Highly recommended for absorbing and cleaning up of any blood drops for leaks that may occur during tube changing.
 - Use for applying antiseptics on infected wounds, applying topical ointments, cleaning cuts or wounds of foreign matter that may have entered the wound and mopping up of blood post invasive procedures such as post injection administration or collection of blood sample.
- **Uses/Instructions:**
 - Used for dressing of open wounds for preventing infection.
 - Generally used for biological sample collection and to avoid sample contamination.
 - Dependable for covering up minor injuries, cuts, grazes, minor open wounds, post surgery or abrasions.
 - Mainly used in non - invasive surgery.
- **Common Materials Used:**
 - Sterile Gauze Swab - 5x5 cm, 7.5x7.5 cm, 10x10 cm.
 - 100% Pure Cotton Yarn Woven.
 - Plain Weave TPI (Threads Per Inch) - 30 x 20 (Warp x Weft).
 - Common Size 47 GSM, individually packed in sterile pouches.
 - pH Level - Neutral to slightly acidic (5.5 – 7.5) yarn - 30s to 40s cotton yarn.



Sizes	Pack Size	MRP
10 cm x 10 cm x 12 ply	1 x 100	₹ 975
7.5 cm x 7.5 cm x 12 ply	1 x 100	₹ 825
5 cm x 5 cm x 12 ply	1 x 100	₹ 675
10 cm x 10 cm x 8 ply	1 x 100	₹ 950
7.5 cm x 7.5 cm x 8 ply	1 x 100	₹ 925
5 cm x 5 cm x 8 ply	1 x 100	₹ 890

THE KEY ATTRIBUTES OF NON STERILE GAUZE SWAB ARE AS FOLLOWS:

- Characteristics/Features:**
 - Janis Care Non Sterile Gauze swab is a soft, strong & durable cotton gauze swab.
 - 100% Cotton. Gauze type –13. White colored. Available in 12 ply and 8 ply.
 - Is highly absorbent with low adhesion properties.
 - Provided with large lids and is double wrapped with a covering to avoid contamination.
 - Ideal for wound dressing or for absorbing blood and body fluids during a surgery.
 - Has a folded edge, low linting and no loose particles.
- Uses/Instructions:**
 - Ideal for wound dressing or for absorbing blood and body fluids during a surgery.
 - Useful for skin cleansing, wound debridement and in dressing packs.
 - Highly useful to clean & cover minor wounds and to absorb exudate.
 - Can be used to absorb blood or exudate during a surgical operation after sterilization.
- Common Materials Used:**
 - Nonsterile Gauze Swab – 5x5 cm, 7.5x7.5 cm, 10x10 cm.
 - 100% Pure Cotton Yarn Woven.
 - Plain Weave TPI (Threads Per Inch) – 30 x 20 (Warp x Weft)
 - Common Size 47 GSM, individually packed in sterile pouches
 - pH Level – Neutral to slightly acidic (5.5 – 7.5) yarn – 30s to 40s cotton yarn.





Product Name	Pack Size	MRP
Gauze Than (90 cm x 18m)	1 x 1	₹ 900
Gauze Than (90 cm x 9m)	1 x 1	₹ 900

THE KEY ATTRIBUTES OF GAUZE THAN BANDAGE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Made up of a roll of soft, woven, cotton cloth that provides of a size (90 cm x 18 meter) which is highly absorbent and breathable making it highly suitable for wound care and surgical procedures.
- **Uses/Instructions:**
 - Highly effective for absorbing exudates and keeping wounds clean and dry, which aids quick wound healing.
 - Useful in hospitals for wiping, padding, dressing of wounds and during the process of surgery.
- **Common Materials Used:**
 - Material 100% Cotton Yarn.
 - Fabric Type Woven, Plain Weave.
 - TPI (Threads Per Inch) – 30 x 20 (Surgical Grade).
 - GSM (Fabric Weight) – Varies by TPI : 40–47 GSM (30x20).
 - Yarn Count 20s, 30s, 40s cotton (combed or carded).
 - Width 90 cm (standard), 100 cm, 120 cm (custom widths available).
 - Length per Than 50 meters, 100 meters, or as per requirement.



Sizes	Pack Size	MRP
15 cm x 5 mtr	1x1	₹ 350
10 cm x 4 mtr	1x1	₹ 275
7.5 cm x 4 mtr	1x1	₹ 250
5 cm x 4 mtr	1x1	₹ 200

THE KEY ATTRIBUTES OF CREPE BANDAGES ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Janis Crepe Bandage B.P. is used to support healing of sprains and strains for providing good compression to injured areas.
 - Excellent elasticity that allows joints and muscles easy flexing.
 - Provides very good compression against inflammation of the knee, ankle and other areas.
 - Made from woven elastic cotton, which is skin friendly and absorbent.
- **Uses/Instructions:**
 - For relieving muscle, joint sprains, strains and to secure a joint or a wound so that the normal functioning can continue.
 - To relieve spasm and reduce swelling.
 - Provides quick relief from pain and inflammation from minor injuries to joints, muscles and supports weak joints.
- **Common Materials Used:**
 - Crepe bandages - 100% Cotton or Cotton blended with Elastic Yarn (e.g., Polyester, Rubber, or Lycra).
 - Crepe Weave (Special warp and weft interlocking for elasticity).
 - Lengthwise elasticity: 60% – 100%.
 - Length - Typically 4 meters (stretched up to 6 meters).
 - 90 – 120 GSM (varies by blend and tension).
 - Edges- Woven or stitched (non-fraying).
 - Fastening Options - Metal clips, Velcro closure, or self-adhesive (optional).



BOX PACKAGING - Plastic

Sizes	Pack Size	MRP
10 cm x 9 mtr	1x1	₹1100

BOX PACKAGING - Tin

Sizes	Pack Size	MRP
10 cm x 9 mtr	1x1	₹1500

THE KEY ATTRIBUTES OF ELASTIC ADHESIVE BANDAGE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Janis Care Elastic Adhesive Bandage is a strong, adhesive, water repellent bandage that provides moderate compression and excellent support during muscle and soft tissue injuries.
 - Is flesh colored, light weight, ideal size bandage which has high flexibility and is easy to wrap.
 - Has non graying strongly bound edges.
 - Is washable with warm water.
- **Uses/Instructions:**
 - Matches the needs of an ideal elastic bandage to meet orthopedic needs as well as a pressure bandage in burns and skin grafts.
 - Excellent to provide protection during sport injuries.
 - Ideal to secure irregularly contoured injured body sites as its stretchable material maintains a firm grip around the injured part to control edema and swelling ensuring body parts movement at the same time such as during dislocation of joints.
 - Useful aid in a post -surgical ward.
 - Essential component of every first aid kit.
- **Common Materials Used:**
 - Elastic Adhesive Bandage Cotton fabric (woven) with embedded elastic yarn (latex or latex-free).
 - Adhesive Type - Zinc Oxide adhesive (skin-friendly, hypoallergenic if required).
 - Stretchability - Approx. 60% – 80% lengthwise stretch.
 - Weave - Plain or crepe weave with longitudinal elasticity.
 - Fabric GSM - 80 – 120 GSM (depending on elasticity and cotton density).
 - Edges - Feathered or straight cut (machine cut to prevent fraying).



Sizes	Pack Size	MRP
10 cm x 10 cm	1 x 80	₹ 5600
10 cm x 20 cm	1 x 40	₹ 3600

THE KEY ATTRIBUTES OF COMBINE DRESSING ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Combines a blend of highly absorbent cotton with rayon fibers.
- **Uses/Instructions:**
 - Janis Care Combine Dressing is used for dressing and treating wounds, especially larger wounds with moderate to heavy bleeding.
 - Read the label carefully before use.
 - Keep in a cool and dry place away from sunlight.
- **Common Materials Used:**
 - Construction Multi-layer dressing - Gauze + Absorbent Core + Optional Backing Layer.
 - Outer Layer (Contact Layer) - Woven cotton gauze or non-woven spunlace depending on product type.
 - TPI (Threads Per Inch) - If gauze is used: 30 x 20 TPI (surgical-grade, 40–45 GSM).
 - Gauze GSM 40–47 GSM (for 30 x 20 TPI).
 - Gauze Yarn Count - 40s or 30s cotton yarn (warp and weft).
 - Absorbent Core - 100% bleached cotton or cellulose fluff pulp (high absorbency).
 - Back Sheet (optional) - Polyethylene (PE) film – fluid-impermeable backing to prevent strike-through
 - Edge Finish Heat - sealed or ultrasonically sealed to prevent linting and leakage.





Product Name	Pack Size	MRP
Janis Care Abdominal MOP	1x10	₹ 1100

THE KEY ATTRIBUTES OF ABDOMINAL MOP ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Highly absorbent Janis Care Abdominal Mop is made up of a cotton gauze fabric capable of soaking up large amount of blood, fluids and other debris.
 - All four edges of the mop are sealed to prevent fraying, lint and leakage.
 - A sturdy loop or tape is stitched at one corner for easy handling and retrieval, preventing it from shifting or falling out of place during surgery.
 - Available in various thicknesses (ply) to accommodate different surgical needs.
 - Built-in X-ray detectable thread to ensure complete removal and prevent retained foreign objects, which can lead to complications like infection or tissue damage.
- **Uses/Instructions:**
 - Janis Care Abdominal Mop is used during surgery as a covering to prevent tissue injury. It acts as an absorbent material, helping to absorb fluids and maintain a clear surgical field.
 - Acts as an absorbent material helping maintain a clear surgical field by absorbing fluids.
 - Protects surrounding tissues from damage during dissection and manipulation.
- **Common Materials Used:**
 - Material – 100% Pure Cotton, bleached surgical-grade gauze.
 - Weave Type – Plain weave gauze.
 - TPI (Threads Per Inch) 45 TPI.
 - GSM (Fabric Weight) 40–47 GSM (for 30x20 TPI), 28–32 GSM (for 19x15 TPI).
 - Yarn Count – 30s or 40s cotton yarn.
 - Edge Finish – Folded edges (to avoid loose threads/linting).
 - Sterility – Sterile (EO or Gamma sterilized), also available in non-sterile.
 - Sterilization Method – Ethylene Oxide (EO) / Gamma Radiation / Autoclave.
 - Radiopaque Thread – Optional: X-ray detectable barium sulfate thread stitched into the mop.



Product Name	Pack Size	MRP
Gamjee Roll (10 cm)	1x1	₹ 430
Gamjee Roll (15 cm)	1x1	₹ 490

THE KEY ATTRIBUTES OF GAMJEE ROLL ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Gamjee roll provides low adherence to the wound, making it easy to remove while offering good absorbency and wound protection.
- **Uses/Instructions:**
 - It's commonly used for wound cleaning as a primary wound dressing or as a protective layer over an existing dressing, for wounds cuts and burns.
 - They are primarily used for wound dressings, especially in medical settings, to absorb fluids and protect the wound from infection and external trauma.
 - The absorbent cotton core helps draw out fluids from the wound, promoting healing.
 - Gamjee Roll offers a cushioning effect and protects the wound from external contamination and trauma.
- **Common Materials Used:**
 - Material Composition – Inner Layer: 100% Absorbent Cotton Wool, Outer Layer: Surgical Gauze (100% Cotton).
 - Gauze TPI (Threads/inch) – 30 x 20 TPI (Standard Surgical Grade) or 19 x 15 (Economy).
 - Gauze GSM (Approx.) – 40–45 GSM (for 30x20 TPI gauze).
 - Cotton Conforming To IP / BP / USP standards, low lint, high absorbency.
 - Sterility/Non-sterile (standard) – can be sterilized by EO/Gamma on request.
 - Width Options - 5 cm, 7.5 cm, 10 cm, 15 cm, 20 cm, 25 cm, 30 cm.
 - Length Options - Standard: 3 meters / 4 meters / 5 meters (customizable).
 - Roll Diameter – Varies based on ply and width (approx. 8–15 cm).
 - Ply / Thickness – 2-ply, 4-ply, 6-ply (depending on usage).
 - Edge Finish – Folded or stitched edges to prevent fraying (optional).



Sizes	Pack Size	MRP
7.5 cm x 9.14 m (3 inch)	1 x 4	₹ 900
5 cm x 9.1 m (2 inch)	1 x 6	₹ 900
2.5 cm x 9.1 m (1 inch)	1 x 12	₹ 900
2.5 cm x 9.1 m (1/2 inch)	1 x 24	₹ 900

THE KEY ATTRIBUTES OF MICROPOROUS PAPER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Microporous Paper is a medical adhesive tape made from hypo-allergic, non-woven, water repellent paper which is 100% latex free with tiny pores allowing it to be breathable and gentle on the skin.
 - It is light weight, comfortable to wear and is easy to tear and leaves no residue on the skin after removal.
 - Easy to tear for quick and convenient application.
- **Uses/Instructions:**
 - Used for anchoring bulky dressing and allows customization of the tape width.
 - Dependable in securing dressings and devices such as I.V. lines and catheters, medical equipment such as electrodes and wires to the skin.
 - Easily conforms to the shape of the body surface.
 - Suitable for taping fragile skin.
 - Read the label carefully before use and keep out of reach of children and away from direct sunlight.
- **Common Materials Used:**
 - Base Material - Cellulose fiber (Non-woven microporous paper).
 - Polypropylene, polyethylene, or PTFE - Known for its flexibility, tear resistance, and high resistance to moisture and water absorption.
 - Adhesive Type - Medical-grade acrylic adhesive, hypoallergenic.
 - Color - White (standard), skin-tone options available (beige/light tan)
 - Adhesive Strength - Medium tack: Secure hold with easy removal.
 - Air & Moisture Permeability - High: allows skin to breathe, reduces maceration risk.
 - Water Resistance - Slightly water repellent but not waterproof.
 - Tensile Strength - Moderate: allows slight stretch, does not tear easily under tens.



Sizes	Pack Size	MRP
15 cm x 2.7 mtr (6 inch)	1 x 10	₹ 300
10 cm x 2.7 mtr (4 inch)	1 x 10	₹ 275

THE KEY ATTRIBUTES OF JANISOCARE POP ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Made from Leno Weave fabric securely coated with Plaster Of Paris powder so as to adhere firmly to the cloth and does not lose efficacy when wetted with water.
 - Quickly hardens when wetted.
 - Does not irritate the skin when it hardens.
 - Can be easily shaped to conform to the injured area.
- **Uses/Instructions:**
 - Primarily used for immobilizing and supporting fractures, sprains and other musculoskeletal injuries.
 - Read the label carefully before use and keep out of reach of children and away from direct sunlight.
 - In fractures used to immobilize broken bones and allow them to heal in the correct position.
 - In Sprains and Dislocations: To support and stabilize the affected joint during the healing process.
 - Post-operative Stabilization: To immobilize joints or limbs after surgery.
- **Common Material used:**
 - Base Cloth - Cotton gauze (open weave).
 - POP Content - 85–90% by weight.
 - Impregnation - Uniform throughout the roll.
 - Initial Setting Time - 2–4 minutes (after dipping in water).
 - Final Setting Time < 8 minutes.
 - Drying Time (Complete cure) - 24–48 hours.



Sizes	Pack Size	MRP
4.5 cm x 6 cm	1 x 50	₹ 120

THE KEY ATTRIBUTES OF AN IV CANNULA FIXATOR ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Offers secure and comfortable fixation with minimum risk of dislodgement and causing infection.
 - Provides an extra strip for proper adhesion even with movement.
 - Easy to remove.
 - Made from highly flexible material to conform to patients body movement and minimize the risk from pressure injuries.
 - Composed of hypoallergenic material to prevent skin irritation.
 - User friendly design that ensures painless removal.
 - Sterile Packaging ensures freedom from contamination, reduces infection risk.
- **Uses/Instructions:**
 - Read the label carefully before use and keep out of reach of children and away from direct sunlight.
- **Common Material Used:**
 - Base Material – Non-woven fabric / Polyurethane film / Foam.
 - Adhesive Type – Hypoallergenic, medical-grade acrylic adhesive.
 - Backing Layer – Paper or film liner, easy to peel.
 - Pre-cut Design – Special shape with slit or notch to accommodate cannula.






Sizes	Pack Size	MRP
10 cm x 10 cm	1 x 1	₹ 380
10 cm x 25 cm	1 x 1	₹ 1022

THE KEY ATTRIBUTES OF TRANSPARENT DRESSING ARE AS FOLLOWS:

- Characteristics/Features:**
 - Janis care Transparent Dressing is flexible and provides an ideal healing condition.
 - Allows visualization of the wound and acts as a barrier against infection.
 - Is sterile, bio-compatible and water proof but permeable to oxygen.
 - Minimizes the risk of skin allergy and irritation.
- Uses/Instructions:**
 - Helps maintain a moist environment, which is crucial for wound healing and the promotion of autolytic debridement (the body's natural process of removing dead tissue).
 - Acts as a barrier against bacteria and other contaminants, preventing infection and promoting a clean healing environment.
 - Allows for visual assessment of the wound without disturbing the healing process.
 - Highly useful in healing surgical incisions, abrasions, skin tears, blisters, skin graft donor sites, superficial burns and pressure ulcers.
 - Used to cover and protect catheter sites, securing devices to the skin.
 - Can be used as protective eye coverings.
 - Assures faster wound healing, reduced pain lesser and scarring.
- Common Material Used:**
 - Film Material - Polyurethane (PU) film – transparent and breathable.
 - Adhesive - Hypoallergenic, medical-grade acrylic adhesive.
 - Backing Liner - Paper or PET liner with easy-peel slit.
 - Sterility- EO (ethylene oxide) or gamma sterilized.
 - Transparency \geq 98% light transmission.
 - MVTR (Moisture Vapor Transmission Rate) \geq 800–1200 g/m²/24hr.
 - Adhesion to Steel (Tack Test) \geq 1.5 N/cm.
 - Thickness of Film 25–35 microns



Surgical Disposal Products

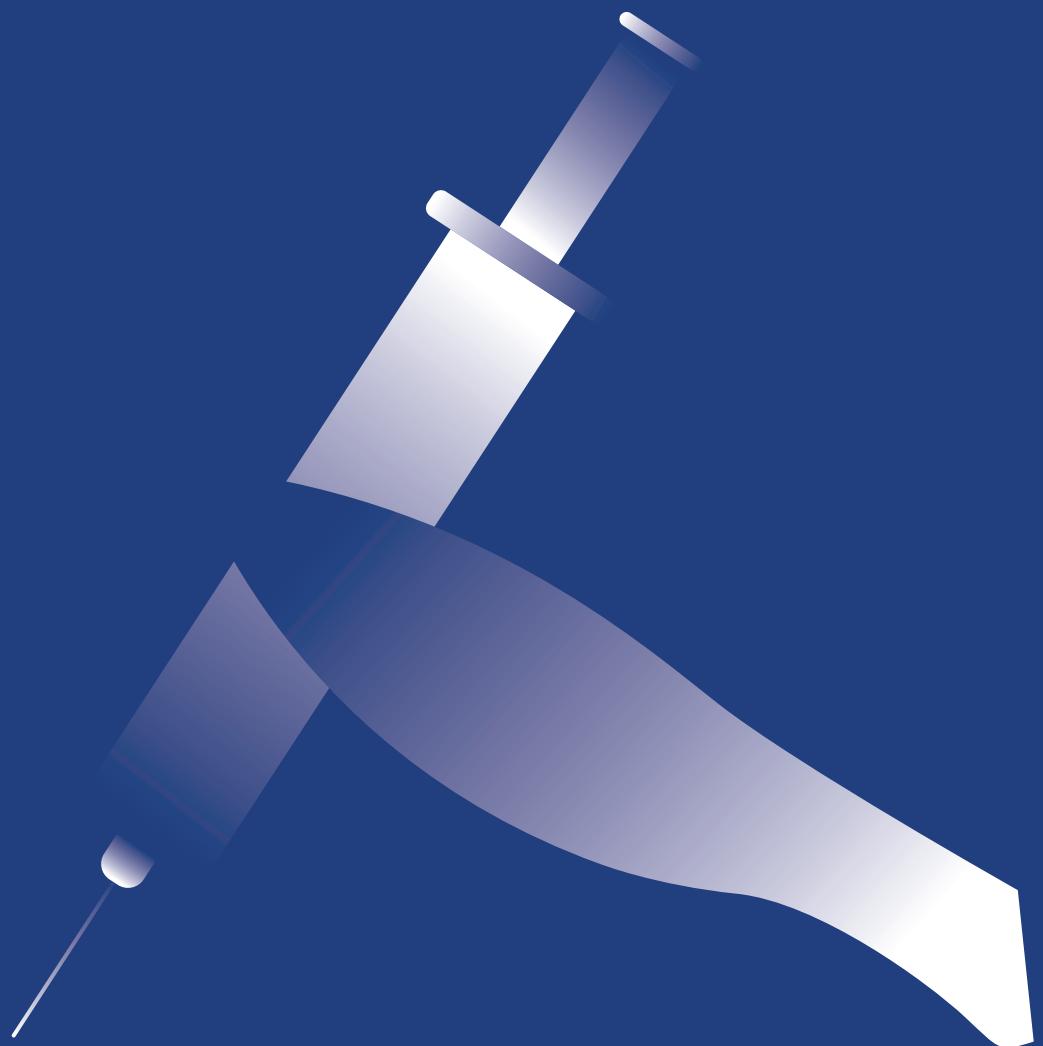



Sizes	Pack Size	MRP
3 ply - 85 GSM Blue Box	1 x 50	₹ 8
3 ply - 85 GSM Black Box	1 x 50	₹ 8
3 ply - 65 GSM Blue (Melt Blown) Box	1 x 50	₹ 8

THE KEY ATTRIBUTES OF A 3 PLY SURGICAL MASKS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Janis Care 3 layered & 5 layered surgical mask, that is comfortable, light, breathable and skin friendly.
 - It is ergonomically designed with a wire to support around the nose area for comfortable fit, equipped with a nose clip and a comfortable ear and head loop.
 - Skin friendly spun bond non - woven, water resistant fabric.
 - Electrostatically charged melt - blown Polypropylene nonwoven fabric.
 - Flat elastic band to ensure no slip page.
 - Flexible G.I. single core nose wire for proper fit.
 - Color - Medical Blue.
 - ISI Approved.
- **Uses/Instructions:**
 - Surgical masks are primarily used as a barrier to protect both the wearer and others from respiratory secretions, such as droplets and aerosols.
 - Help reduce the risk of infection transmission in healthcare settings and during public health events.
 - Patients may also wear surgical masks to prevent the spread of their own respiratory droplets and aerosols to others.
- **Common Materials Used:**
 - General product Description & Specifications – 3 Layer with ear loop, With Flexible Nose wire, Latex free Polyester Elastic and Highly breathable and comfortable Non-Woven PP Fabrics.
 - Raw materials (BOM).
 - Layer 1: Non-woven Spun bond – 35 GSM (Medical Blue).
 - Layer 2: Melt Blown 25 GSM.
 - Layer 3: Non-woven Spun bond- 25 GSM (White).
 - Ear loop: 3mm Polyester Elastic – 165 MM.
 - Nose wire: PP Insulated GI Nose Wire 100 MM.



Product Name	Pack Size	MRP
N 95 Mask	1x1	₹ 45

THE KEY ATTRIBUTES OF AN N95 MASK ARE AS FOLLOWS:

- **Characteristics/Features:**
 - N95 respirator is a respiratory protective device designed to achieve a very close facial fit and very efficient filtration of airborne particles.
 - Made up of 5 layers. Outermost and innermost skin friendly layers are made up of spun bonds to trap large sized particles. Second and fourth layer made up of electrostatically charged Melt blown material, traps extremely fine particles whereas the middle (third) layer is of non-woven needle punch material that gives cushioning effect and lends comfort to the wearer of the mask.
 - Fits snugly around the nose thereby preventing the air leakage from around the corners.
- **Uses/Instructions:**
 - It is a single-use, disposable respiratory protective devices used and worn by healthcare personnel during procedures (within 6 feet) to protect both the patient and the healthcare personnel from transfer of microorganism, body fluids and particulate material.
 - Designed to filter out 95% of the air suspended particle that traps bacteria, viruses, air borne dirt and harmful air pollutants.
 - Effective in filtering out particles as small as 0.3 microns including bacteria and viruses.
 - Allows the wearer to breath freely.
 - Is fluid resistant against the sprays and splashes.
- **Common Material Used:**
 - Outer Layer – Spunbond polypropylene (hydrophobic).
 - Middle Layer – Melt-blown polypropylene (filtration layer).
 - Inner Layer – Soft spunbond for comfort.
 - Nose Clip – Aluminum strip (adjustable, embedded or external).
 - Ear Loops/Head Straps-Elastic bands (latex-free).



Shoe Cover

Product Name	Pack Size	MRP
Shoe Cover	1 x 50 pairs	₹ 7

Cap

Product Name	Pack Size	MRP
Shower Cap (Surgeon/Male)	1 x 100 pcs	₹ 7
Bouffant Cap (Female)	1 x 100 pcs	₹ 5

THE KEY ATTRIBUTES OF SHOE COVER & CAP ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Is waterproof, non-skid, comfortable, disposable, breathable and lint-free to prevent contamination in sterile environments.
 - Ensures snug fit, preventing the cover from slipping off and potentially exposing the wearer to contaminants.
 - Is ultrasonically sealed to ensure sterility and prevent contamination.
 - Produced without manual handling.
 - Strong. Withstand wear and tear.
- **Uses/Instructions:**
 - Prevent contamination and maintain a sterile environment.
 - Act as a barrier between the wearer's shoes and the floor.
 - Reduces the risk of spreading dust, dirt and other contaminants.
 - Prevent the transfer of microorganisms from the outside to the clean or sterile areas, thus reducing the risk of infection.
- **Common Material Used:**
 - Non-Woven PP – Polypropylene (25–40 GSM), breathable, lightweight.
 - Plastic (PE/CPE)- Chlorinated polyethylene / polyethylene – waterproof.



Gloves	Pack Size	MRP
Latex Examination 6 gms (Powder Free)	1 x 100	₹ 1650
Examination Latex 6 gms (Powdered)	1 x 100	₹ 1600
Examination Nitrile	1 x 100	₹ 1450
Examination Nitrile Gloves	1 x 20	₹ 450
Latex Examination Gloves	1 x 20	₹ 400
Sterile Surgical Latex Gloves	1 x 2	₹ 68

THE KEY ATTRIBUTES OF EXAMINATION GLOVES ARE AS FOLLOWS:

- **Latex Gloves:**
 - Offers high level of comfort, elasticity and natural bio degradability.
 - Highly suitable for carrying out medical procedures with a natural feel for carrying out tasks requiring motor skills and dexterity.
 - Prevents cross contamination between the care giver and the patients and causes minimum fatigue during extended use.
 - Excellent elasticity ensures snug fit.
 - Provide protection against viruses, pathogens, bacteria and certain chemicals, offering a barrier against contamination.
- **Nitrile Gloves:**
 - Provide exceptional strength, durability and chemical resistance.
 - Good flexibility and comfortable wear ensures precise handling.
 - High puncture resistance allows use against sharp objects and safety against hazardous material.
- **Uses/Instructions:**
 - Offer strong protection against chemicals, pathogens and physical hazards.
 - Prevent the spread of infections and protect healthcare professionals from exposure to blood and other bodily fluids during exams and procedures.
 - Insert your hand into the gloves. Stretch your fingers to achieve a snug fit.
- **Common Material Used:**
 - Parameter - Specification.
 - Length - Minimum 240 mm (ASTM/EN standards).
 - Palm Width (M Size) - 95 ± 10 mm.
 - Thickness (at fingertips) - 0.08–0.15 mm (varies by material).
 - Weight - Latex: 4–6g, Nitrile: 3–5g, Vinyl: 4–5g (per glove).
 - Color Latex - Natural / White, Nitrile: Blue / Purple, Vinyl: Clear.



Product Name	Pack Size	MRP
Transducer Protector	1x1	₹ 464

THE KEY ATTRIBUTES OF A HEMODIALYSIS TRANSDUCER PROTECTOR ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Shields sensitive electronic components from damage and contamination, primarily in medical applications like hemodialysis.
 - A Sterile, Non pyrogenic hydrophobic membrane with a small pore size (typically 0.2 microns) to allow gas exchange while preventing the passage of fluids and bacteria.
- **Uses/Instructions:**
 - Primarily used in hemodialysis to prevent contamination of the machine and pressure monitoring systems by blood and dialyzing fluids.
 - Act as a barrier between the patient's blood and the dialysis machine, ensuring safe and accurate pressure readings.
 - Protect patients from potential bloodborne infections and improve the overall safety of the dialysis process.
 - It isolates the patient's blood from the dialysis machine, preventing accidental contamination of the machine's internal components.
- **Common Material Used:**
 - Parameter - Specification.
 - Diameter - 25–35 mm (standard).
 - Height - 15 – 25 mm.
 - Weight - Approx. 3–5 grams.
 - Connection Ports - Female-to-male or male-to-male Luer lock.
 - Filter Area ~2–5 cm² (hydrophobic membrane).
 - Flow Direction - Unidirectional or bidirectional (as per model).



Product Name	Pack Size	MRP
Janis Blood Tubing Set	1x1	₹ 650

THE KEY ATTRIBUTES OF A BLOOD TUBING SET ARE AS FOLLOWS:

- Characteristics/Features:**
 - Dialysis blood tubing sets post-pump, also known as post-pump bloodlines, are disposable, sterile devices used to transfer blood from the dialyzer back to the patient after it has been processed by the pump.
 - Primary Volume 94ml arterial, 62ml venous.
 - Blood path length: 152" (385 cm) arterial 16" (295cm) venous.
 - Bloodline path tubing (arterial and venous) 0.180 (4.57 mm) I.D., 0.265" (6.73 mm) O.D.
 - Bloodline pump segments: 0.317" (8.0 mm) I.D., 0.474" (12.00 mm) O.D. and 13.5 (34.3 cm) length.
 - Bloodline provided with colour coded components: Red on the arterial side and Blue on the venous side.
 - Attached by priming set with Y- injection site.
- Uses/ Instructions:**
 - It ensures the efficient and safe flow of blood during treatment, acting as a bridge between the patient's body and the external dialysis system.
 - The set connects the patient's vascular access point (usually an AV fistula) to the dialysis machine, allowing for blood to be drawn out, filtered and returned.
- Common Material Used:**
 - Material - Medical-grade, non-toxic PVC (DEHP-free optional).
 - Components - Arterial line + Venous line.
 - Color Coding - Red (arterial), Blue (venous).
 - Pump Segment Compatibility - Standard peristaltic pump on dialysis machines.
 - Tubing Material Medical - grade PVC.
 - Tubing Diameter - 6.0–6.5 mm ID (pump segment); 3.0–4.5 mm ID (others).
 - Length Approx - 150–250 cm (varies by brand/model).
 - Blood Flow Rate Supports 200–500 mL/min.



Product Name	Pack Size	MRP
Vented	1x1	₹ 310

Product Name	Pack Size	MRP
Non - Vented	1x1	₹ 310

THE KEY ATTRIBUTES OF I.V. (INFUSION SET) ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a sterile medical device used to deliver fluids, medications and blood products directly into a patient's bloodstream, includes a drip chamber for fluid visualization, kink-resistant tubing for smooth flow and a roller clamp for precise flow rate adjustment.
 - Incorporates a transparent, flexible chamber where fluid flows one drop at a time, allowing for easy visualization of flow rate and detection of air bubbles.
 - A fluid filter is located in the drip chamber, typically 15 micron, to remove particulate matter and contaminants from the fluid.
- **Uses/Instructions:**
 - Used to deliver fluids, medications and nutrients directly into a patient's bloodstream through a vein.
 - Used for giving treatments, including hydration, medication administration, blood transfusions and nutritional support.
 - IV sets are vital for providing nutritional support to patients who are unable to eat or have difficulty absorbing nutrients through the digestive system.
- **Common Material Used:**
 - Tubing - Medical-Grade PVC (Polyvinyl Chloride).
 - Drip Chamber - Soft PVC or Polypropylene (PP).
 - Spike - Polypropylene (PP).
 - Air Filter (vented sets) - Hydrophobic membrane filter.
 - Fluid Filter - Nylon / Polyester / Polypropylene mesh.
 - Roller Clamp - Polycarbonate (PC) or ABS (Acrylonitrile Butadiene Styrene) Plastic.
 - Injection Port / Y-siteLatex-free Rubber or Thermoplastic Elastomer (TPE).
 - Luer Connector - Polypropylene (PP).
 - Protective Caps - Polyethylene (PE).
 - Optional Needle - Stainless Steel (21G or 23G).



Product Name	Pack Size	MRP
B.T Set (150 cm)	1x1	₹ 310

THE KEY ATTRIBUTES OF BLOOD TRANSFUSION SET ARE AS FOLLOWS:

- Characteristics/Features:**
 - A blood transfusion set is a medical device used to deliver blood or blood components to a patient intravenously.
 - It is designed for safe and controlled transfusion, utilizing gravity and a series of components to ensure accurate flow and prevent contamination.
 - Clamp: A roller clamp or other device to regulate and control the flow rate of the blood.
- Uses/Instructions:**
 - Red Blood Cell Transfusions.
 - Used to treat anemia or restore blood volume following significant blood loss.
 - Platelet Transfusions: Used to treat thrombocytopenia (low platelet count) and help control bleeding.
 - Plasma Transfusions: Used to replace plasma, which is involved in blood clotting and can be necessary for patients with liver disease or certain coagulation disorders.
 - Whole Blood Transfusions: While less common, can be used to replace all blood components in cases of severe blood loss.
- Common Material Used:**
 - Spike - Medical-grade ABS (Acrylonitrile Butadiene Styrene) / PP.
 - Drip Chamber - Soft PVC or Polypropylene.
 - Blood Filter - Nylon / Polyester Mesh.
 - Tubing - Medical-grade PVC.
 - Roller Clamp - ABS / Polycarbonate.
 - Injection Port (Y-site) - Latex-free Rubber / TPE.
 - Luer Connector - PP - Luer Lock / Slip.
 - Protective End Caps - PE or PP.
 - Needle (optional) - Stainless Steel (21G/23G).



Syringe

Sizes	MRP
2 ml	₹ 25
3 ml	₹ 28
5 ml	₹ 39
10 ml	₹ 51
20 ml	₹ 60
50 ml	₹ 75

Needle

Sizes	MRP
16 g	₹ 5
18 g	₹ 5
26 g	₹ 5
24 g	₹ 5
25 g	₹ 5

THE KEY ATTRIBUTES OF SYRINGES & NEEDLES ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Sterile, designed for single use, accurate in volume measurement and easy to use.
 - Made up of high quality polypropylene to ensure effective medicine delivery/blood collection. Latex free elastomer gasket to minimise the allergic reactions.
 - Cutting edges coated with silicon to minimise pain. Smooth penetration causing minimum discomfort to the patient. Color code identification in needles as per ISO : 6009 standard for ease of use.
- **Uses/Instructions:**
 - Used to inject medication or draw blood samples and to deliver medications via intramuscular, subcutaneous or intradermal injection and to collect blood samples for laboratory tests and blood donations. Also used in procedures like allergy testing, tuberculin testing and administering local anesthetics.
- **Common Material Used:**
 - Single-use, disposable medical syringes.
 - Material: Generally polypropylene for barrel, plunger components made of plastic with rubber/seal for tight fitting.
 - Sterilization: Many come sterile, often Ethylene Oxide (EtO) sterilized.
 - 1. Luer-Lok tip (secure twist-on needle/hub).
 - 2. Luer slip tip.
 - Catheter tip (for flushing, feeding tubes etc.)
 - Volumes available: 1 mL, 3 mL, 5 mL, 10 mL, 20 mL, 30 mL, 50 mL etc.
 - Graduations / scale markings: clearly marked major and minor graduations for accurate dose measurement.
 - Wasted space ("dead space") minimized for some models.



Products Name	Pack Size	MRP
Spinal Needle (23 No.)	1x1	₹ 185
Spinal Needle (25 No.)	1x1	₹ 185
Spinal Needle (26 No.)	1x1	₹ 185
Spinal Needle (27 No.)	1x1	₹ 185

THE KEY ATTRIBUTES OF SPINAL NEEDLES ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Easy to use, minimize the risk of complications like post-dural puncture headache (PDPH), and allow for rapid and accurate cerebrospinal fluid (CSF) detection and flow.
- **Uses/Instructions:**
 - Used to inject analgesics/anaesthetics directly into the CSF usually at a point below the second lumbar vertebra.
 - Used for spinal anesthesia, lumbar punctures and collecting cerebrospinal fluid (CSF) samples.
- **Common Material Used:**
 - Available in gauges 18 G to 27 G.
 - Available in lengths ranging between 90mm to 178mm (≈3.5 in to 7 in) depending on gauge/type.
 - Bevel type: Quincke bevel.
 - Material: Stainless steel cannula.
 - Hub material: Usually polypropylene; hub type is luer hub.
 - Sterilization: Ethylene Oxide (EO) sterilized.
 - Shelf life: ~5 years under specified storage conditions (e.g. around 25°C ±2°C) unless otherwise noted.
 - Single-use, disposable.



IV Cannula	Pack Size	MRP
18 No. - Green	1x1	₹ 287
20 No. - Pink	1x1	₹ 287
22 No. - Blue	1x1	₹ 287
24 No. - Yellow	1x1	₹ 287

THE KEY ATTRIBUTES OF I.V. CANNULA ARE AS FOLLOWS:

- **Characteristics/Features:**
 - I.V. cannula is a sterile, single-use and designed for ease of insertion with minimal pain usually in the hand or back of the hand.
 - Has a clear flashback chamber, a needle shielding mechanism and flexible wings for secure fixation.
 - Is sterile until use and designed for single - patient use to minimize the risk of infection.
- **Uses/Instructions:**
 - It facilitates efficient medication administration, allows for reliable fluid delivery, enables blood sampling and minimizes the need for repeated needle insertions.
 - It enhances the speed and efficiency of treatment by bypassing the digestive system.
 - Enables quick and efficient delivery of medications directly into the bloodstream, ensuring rapid absorption and response.
 - It is essential for blood transfusions, providing a safe and efficient way to transfer blood and blood products.
- **Common Material Used:**
 - Catheter - Medical-grade PTFE or FEP (Fluorinated Ethylene Propylene).
 - Needle Stainless Steel (AISI 304 or 316L).
 - Hub Polypropylene (PP).
 - Flashback - Chamber Polycarbonate (PC).
 - Protective Cap - Polyethylene (PE).
 - Injection Port - Latex-free synthetic rubber or TPE.
 - Valve (if present)Silicone or medical-grade elastomer.



Product Name	Pack Size	MRP
3-Way Stop Cock	1x1	₹ 320

THE KEY ATTRIBUTES OF THREE WAY STOP COCK ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Has a uniform and fixed dead space for accurate drug administration.
 - Smooth rotation of the tap over 360° is possible. Designed to ensure smooth flow of blood at high pressures. Ensures quick stoppage or starting of the fluid flow.
 - Allows healthcare professionals to administer and manage multiple medications or fluids efficiently.
- **Uses/Instructions:**
 - Used for precise and accurate dosing of medications directly into a patient's IV line.
 - Infusing Fluids: It is used to infuse various fluids, including saline, electrolytes or medications, into a patient's bloodstream.
 - It enables healthcare professionals to withdraw blood samples from a patient's IV line without multiple needle insertions.
 - Three-way stop cocks can be used in pressure infusion and blood pressure monitoring procedures.
 - Respiratory Therapy: The male end of the stopcock can fit into the pilot balloon of an endotracheal tube, allowing for control of the cuff inflation/deflation.
 - Instruction : Remove it from its packaging, ensuring sterility and that the valve is closed. Connect one or two IV lines to the female ports as needed. Remove the cap from the male port and connect the IV cannula securely. Open the valve to allow any air to escape, then close the valve. Rotate the valve to control the direction of fluid flow between the ports. Dispose of the stopcock safely after use.
- **Common Material Used:**
 - Body – Medical-grade Polycarbonate (PC).
 - Handle/Stopcock Tap Polyethylene (PE) or ABS Plastic.
 - Caps & Luer Connectors Polypropylene (PP).
 - Gasket / Seal Medical – grade silicone or TPE (latex-free).
 - Extension Tube (optional) DEHP-free PVC, approx. 10–100 cm.





Product Name	Pack Size	MRP
Plain Sheet	1x1	₹ 395

THE KEY ATTRIBUTES OF PLAIN SHEET ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Disposable, light weight, blue polysheet 100 cm x 150 cm (pack of 10 pieces in a contamination proof disposable pack).
 - Used to protect neonates from catching infections.
- **Uses/Instructions:**
 - Used in hospitals, nursing homes, by GP's and Dentists in their clinics.
- **Common Material Used:**
 - Type - Disposable Surgical Plan Sheet / Bed Sheet.
 - Usage - Single-use only.
 - Sterility - Sterile or Non-Sterile (as per requirement).
 - Dimensions (LxW) - 120x210 cm, 100x200 cm, or customizable.
 - Material Type - SMS / SMMS / Spunbond Non-Woven Fabric.
 - GSM (Grams/m²) 25-45 GSM (typical).
 - Color - Blue, Green, or White.





Product Name	Pack Size	MRP
Underpads	1 x 1	₹ 250

THE KEY ATTRIBUTES OF UNDERPADS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Also called as Chux pads or bed pads are heavy duty absorbent pads that help protect furniture, mattresses, wheel chairs and other surfaces against daytime or spoilage by urine through incontinence.(bed wetting)
- **Uses/ Instructions**
 - Underpads are absorbent pads used to protect surfaces from soiling, primarily by absorbing fluids like urine or spills. They are commonly used to manage incontinence, protect bedding, furniture and provide comfort.
 - Secure (if needed): Some underpads have adhesive strips or wings to help secure them in place.
 - Dispose of or launder: Dispose of disposable pads properly. Launder washable pads according to instructions.
 - Underpads are versatile and can be used in various situations: Incontinence management: Protect bedding and furniture from leaks and spills.
- **Common Material Used:**
 - GSM – 55 GSM.
 - Absorbent Core / Technology Construction of internal layers – number of layers, type of material, fluff pulp, SAP, binding, embossing etc.
 - Total weight (per pad): ~ 30–80 g for mid-/large sizes. Heavier pads have better absorbent core and SAP weight.
 - SAP Content (Super Absorbent Polymer) – Determines how much liquid can be absorbed and locked away.
 - Backsheet / Waterproof Layer – Prevents fluid from leaking out of the bottom.
 - Top Sheet Material – Surface layer that touches skin – must be soft, fast-drying, breathable.
 - Edge Sealing / Leak Prevention.
 - Prevents fluid leakage from sides. Quilting helps distribute fluid evenly.



Product Name	Pack Size	MRP
Antiseptic Bath Wipes	1 x 10	₹ 375

THE KEY ATTRIBUTES OF BATH WIPES ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Antiseptic bath wipes are soaked in a highly effective antiseptic Chlorhexidine.
 - High quality spinlace wipes for ergonomic application imbibed with Vitamin E and Aloe Vera Extract.
- **Uses/Instructions:**
 - Used for personal hygiene and cleansing when a full bath or shower is not available. They are disposable, moistened cloths designed to clean and refresh the body.
 - Bath wipes can be used for various purposes, including cleansing after activity, travel or when a full bath is inconvenient.
 - Medical settings: Bath wipes can be used in hospitals or nursing homes to provide patients with a convenient way to clean.
 - Disinfection: Some bath wipes contain antimicrobial agents to help kill germs.
 - Instructions : Open the packaging: Locate the opening and carefully tear or peel open the packet.
 - Remove a wipe: Pull out a single wipe from the package.
 - Clean the desired area: Wipe the body part or area you wish to clean.
 - Discard the wipe: Do not flush bath wipes down the toilet.
- **Common Material Used:**
 - Wipe Fabric - Spunlace non-woven viscose/polyester blend.
 - Base Solution - Purified water (aqua), mild surfactants, skin conditioners.
 - Moisturizers - Glycerin, Aloe Vera, Vitamin E.
 - Preservatives - Paraben-free, non-irritant preservatives.
 - Fragrance - Hypoallergenic (optional).



Product Name	Pack Size	MRP
Cord Clamp	1x1	₹ 45

THE KEY ATTRIBUTES OF CORD CLAMP ARE AS FOLLOWS:

- Characteristics/Features:**
 - Sterile Cord clamp is a medical device designed to secure the umbilical cord after birth, primarily to stop bleeding and allow for the cord to dry and fall off.
 - Has a locking mechanism, a finger grip for safe handling and grooves to prevent slippage.
- Uses/Instructions:**
 - To use an umbilical cord clamp, ensure two clamps are placed on the cord, preferably after pulsation has stopped (delayed clamping). Cut the cord between the clamps with sterile scissors or a blade, and allow the cord stump to dry and fall off naturally. Keep the stump clean and dry during this time.
- Detailed Steps:**
 - Prepare: Open the package and remove the clamp. Wash hands thoroughly, put on gloves and apron.
 - Delayed Clamping (Recommended): Wait for the cord to stop pulsing before clamping, as this can improve the baby's iron stores.
 - Clamp Placement: Place two clamps on the umbilical cord, ensuring they are secure and cannot be opened by hand.
 - Cutting the Cord: Use sterile scissors or a blade to cut the cord between the two clamps. Cut away from the baby's belly.
 - Post-Cut Care: Dab off any excess blood. Allow the cord stump to dry and fall off naturally, which usually takes a week or two. Keep the stump clean.
- Common Material Used:**
 - Clamp Body – Medical-grade Polypropylene (PP) or Polyoxymethylene (POM).
 - Color – Blue .
 - Latex Content – Latex-free.

***GST will be applicable on the rates of all the products.**





Antiseptics & Disinfectants Products



Janisodine - 10% w/v

Sizes	MRP
100 ml	₹ 115
500 ml	₹ 576
2 ltr	₹ 2300

Janisodine - 5% w/v

Sizes	MRP
100 ml	₹ 48
500 ml	₹ 240
2 ltr	₹ 963

THE KEY ATTRIBUTES OF WOUND CARE - 5% & 10% ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a highly effective topical antiseptic for treating and preventing infections prior to a surgery and clinical intervention.
 - It is also used to eradicate infection from the wounds and bed sores.
 - It works by killing a wide range of microorganisms, including bacteria, fungi, protozoa and spores.
- **Uses/Instructions:**
 - It is used for various purposes, including wound care, surgical site preparation and as a hand antiseptic.
- **Composition:**
 - Povidone Iodine (5%)W/V.
 - Povidone Iodine (10%)W/V.



Janisodine – 7.5% w/v

Sizes	MRP
100 ml	₹ 199
500 ml	₹ 996
2 ltr	₹ 3987

THE KEY ATTRIBUTES OF WOUND CARE SCRUB - 7.5% ARE AS FOLLOWS:

- **Characteristics/Feature:**
 - Janisodine scrub 7.5% Surgical Scrub is an antiseptic and disinfectant agent.
 - Highly effective for the treatment and prevention of infections in wounds and cuts.
 - Counteracts infection through bacteria and fungi in wounds, cuts and abrasion injuries.
 - Povidone Iodine may cause side effects like skin irritation, redness or itching at the site of application. If these side effects persist for a longer duration. Consult your physician.
 - Avoid direct contact with eyes. In case of direct contact, wash eyes with water and seek immediate medical attention. Stop using if you have any signs of an allergic reaction.
- **Uses / Instructions:**
 - It is primarily used as a surgical scrub and antiseptic to prepare the skin before surgical procedures. It is also used for pre-operative and post-operative skin cleansing, surgical hand scrubbing and wound care to prevent infections.
 - Pre-operative Skin Cleansing: Helps to cleanse and disinfect the skin before surgery. Surgical Hand Scrubbing: Used to thoroughly clean the hands and forearms of surgical staff before procedures.
 - Wound Care: Can be used to clean and disinfect wounds to prevent infection and promote healing. Pre- and Post-operative Care: Used for general skin cleansing before and after surgical procedures. Antimicrobial Agent: Effective against a wide range of microorganisms, including bacteria, fungi, viruses and other microbes.
- **Composition:**
 - Povidone Iodine: (7.5% w/v)



Products	MRP
50 ml	₹ 37
100 ml	₹ 70
500 ml	₹ 190
1000 ml	₹ 360

THE KEY ATTRIBUTES OF ANTISEPTIC SOLUTION ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is an antiseptic liquid to prevent and treat infection by bacteria, fungi and viruses.
 - Excellent first aid for dabbing on wounds, cuts and abrasions.
 - It is dermatological tested and recommended by the Indian Medical Association for personal hygiene.
- **Uses/Instructions:**
 - It is a powerful antiseptic combination used for cleaning and disinfecting wounds, skin and surfaces.
 - It is a combination of Chlorhexidine which is a broad-spectrum antiseptic that effectively kills or inhibits the growth of many bacteria, fungi, some viruses and Cetrimide, a cationic surfactant with antiseptic properties, often used to enhance the effectiveness of other antiseptic.
 - **Wound Cleansing and Disinfection:** The solution is used to clean and disinfect wounds, cuts, abrasions and burns.
 - **Infection Prevention:** It helps to prevent infections in minor injuries and wounds.
 - **Skin Disinfection:** It can be used for general skin disinfection in medical settings, such as before surgery.
 - **Surgical Hand Scrub:** It is used in surgical hand scrubs to reduce the number of microorganisms on the skin.
- **Composition:**
 - Chlorhexidine Gluconate (0.3%v/v) solution.
 - Cetrimide solution (0.6 %v/v)
 - Purified water.



Products	MRP
50 ml	₹ 45
100 ml	₹ 70
500 ml	₹ 250
1000 ml	₹ 450

THE KEY ATTRIBUTES OF ANTISEPTIC SOLUTION ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is an antiseptic liquid to prevent and treat infection by bacteria, fungi and viruses.
 - Excellent first aid for dabbing on wounds, cuts and abrasions.
 - It is dermatologically tested and recommended by the Indian Medical Association for personal hygiene.
 - It is a combination of Chlorhexidine which is a broad-spectrum antiseptic that effectively kills or inhibits the growth of many bacteria, fungi, some viruses and Cetrimide, a cationic surfactant with antiseptic properties, often used to enhance the effectiveness of other antiseptic.
- **Uses/Instructions:**
 - Wound cleaning and disinfection: It cleans and disinfects wounds, helping to prevent infection. Dilute 1 part of CLENS CARE PLUS in 15 parts of clean water and use.
 - Skin cleansing: It can be used to cleanse the skin, especially before procedures or to prevent infection.
 - Disinfecting surfaces: It can be used to disinfect hard surfaces like floors, tables and countertops.
 - Bathing water disinfectant: It can be added to bathing water to cleanse the skin. Dilute 15 ml (1 tablespoonful) in a bucket of water or in a mug (200ml) water for use as a disinfectant.
 - Personal hygiene: It can be used for personal hygiene, such as after contact with potentially infected surfaces or areas, before or after shaving etc add 15ml (1 tablespoonful) to a mug (200ml) of water.
 - For disinfecting clothes add a spoonful (5ml) of water in half a litre of water.
- **Composition:**
 - Chlorhexidine Gluconate (1.5%v/v) solution.
 - Cetrimide solution (3% v/v)
 - Purified water.



Product Name	Pack Size	MRP
Janzyme	1 x 1 ltr	₹ 4750

THE KEY ATTRIBUTES OF JANZYME ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Highly effective triple enzymatic cleaner that acts against proteins, carbohydrates and fats, hence has an enhanced cleansing action.
 - Low foaming, Non toxic & Bio-degradable.
 - Equally effective against molds and mildew infestation.
 - Recommended for linen, fabric, clothes, upholstery and carpet cleansing. Removes blood stains and removes odour while cleansing.
 - Suitable for addition into manual, automatic and ultrasonic cleansing machines.
- **Uses/Instructions:**
 - Dilution and direction for use: Mix 16 ml (approx.: 1 Tablespoonful) of solution in 984ml (approx.: 1 Litre) of lukewarm RO/DM water and soak the material to be washed/cleansed.
 - Prepare a fresh solution for each application.
- **Precaution:**
 - For external use only.
 - Avoid direct contact with eyes.
 - In case of eye contact flush eyes with fresh water repeatedly till irritation subsides.
 - Keep out of reach of children & seek medical help in case of an emergency.
- **Composition:**
 - Contains multiple enzymes such as Protease, Amylase, Lipase, Cellulase and non ionic surfactants, chelating agents, rust inhibitors & stabiliser. (3% v/v)
 - Purified water.



Sizes	Pack Size	MRP
100 ml	1 x 100 ml	₹ 225
500 ml	1 x 500 ml	₹ 350
5 ltrs	1 x 5 ltrs	₹ 4500

THE KEY ATTRIBUTES OF JANIS SHIELD ARE AS FOLLOWS:

- **Characteristics/Features:**
- Contains a combination of Isopropyl alcohol - a common powerful antiseptic and disinfectant, N-propanol, acts as a disinfectant and solvent and Ethyl hexadecyl-dimethyl ammonium ethyl sulphate - is known for its antibacterial properties.
- **Uses/Instructions:**
- Commonly used as a hand rub for sanitisation prior to surgery or handling surgical instruments or touching a patient.
- Disinfectant and Antiseptic: Widely used in medical settings, homes and industrial environments to disinfect surfaces and as a skin antiseptic for hand sanitization.
- **Composition:**
- Each 100 gm contains:
- Isopropyl alcohol I.P. (2 Propanol).
- N-Propanol (1 Propanol).
- Ethyl-Hexadecyl-Dimethyl Ammonium-ethyl sulphate and other skin protecting substances.



Product Name	Pack Size	MRP
Rust Remover	1x1	₹ 3500

THE KEY ATTRIBUTES OF RUST REMOVER ARE AS FOLLOWS:

- **Characteristics/Features:**
- Highly effective in removing rusty stains from surgical instruments, trays and carts.

- **Uses/Instructions:**
- Contains concentrated corrosive mineral acids hence needs to be handled very carefully.
- Required for rust removal in operating rooms, Outpatient surgical ward, Dental and other departments where minor surgeries are done for derusting surgical instruments and equipment.



Product Name	Pack Size	MRP
Part A + Part B	1x 50 ltr	₹ 3240

THE KEY ATTRIBUTES OF HEMODIALYSIS CONCENTRATE SOLUTION ARE AS FOLLOWS:


- Characteristics/Features:**
 - Specially formulated to have specific electrolyte concentrations that, when diluted, mimic the composition of normal plasma.
 - Specifically designed to facilitate the removal of waste products and excess fluids from the blood during dialysis, while also helping to maintain electrolyte balance and acid-base balance.
 - The composition of the final dialysate reaching the dialyser is a proportioned mixture of Acid concentrate (A Conc), Bicarbonate Concentrate (B Conc) and RO (Reverse Osmosis water).
- Uses/Instructions:**
 - Use Part -B with Part -A in Hydrogen Carbonate Mode of Dialysis Machine:

Hemodialysis Solution	Part -A	Part -B	RO Water	Automatically adjusted
Ratio 36.83	x1ml	1.83 ml	34 ml	by Dialysis Machine



Medical Device



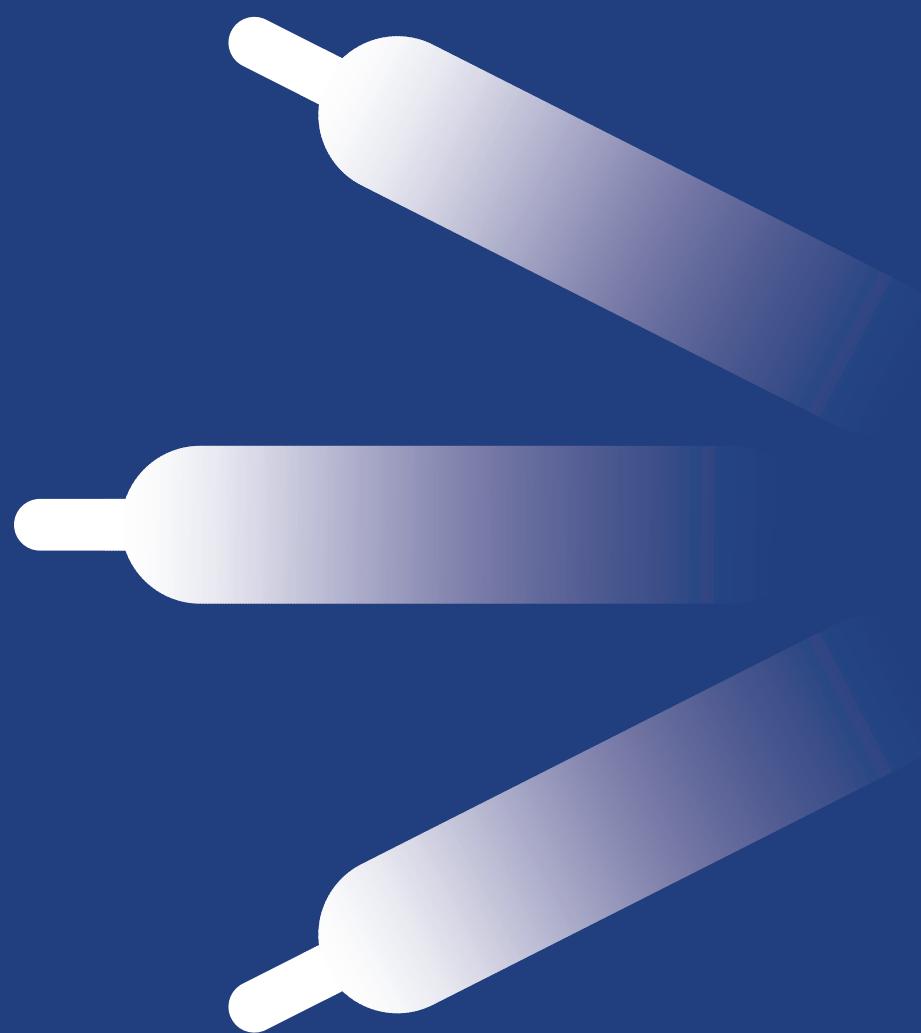



Product Name	Pack Size	MRP
Janis U Safe Forte	1x1ltr	₹ 5600

THE KEY ATTRIBUTES OF U SAFE FORTE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Contains a surfactant and biocide. It has applications as a disinfectant. Also contains trialkyl ethoxy ammonium propionate used as surfactants in cleaning agents, personal care products and as active ingredients in antimicrobial pesticides.
 - Effective non corrosive, non toxic disinfectant for Endoscopes & all types of surgical instruments and is preferred to toxic aldehyde.
 - Is bio-degradable compared to other disinfectants like aldehyde and phenols.
 - Contains 2.2 gms of Dodecyl dipropylene triamine and 1.7 gm of trialkyl ethoxy ammonium propionate.
 - Also contains tensides (other surfactants).
- **Composition:**
 - Dodecyl dipropylene triamine 2.2 gm.
 - Trialkyl ethoxy ammonium propionate.
 - Tensides and Auxillaries.
- **Uses/ Instructions:**
 - Algaecide and Algistat: Dodecyl Dipropylene Triamine is highly effective as an surgical instruments sanitiser as it is an algaecide and algistat, helping control and prevent the growth of algae in various water systems. It ensures clean and clear water environments.
 - Disinfectant and Sanitizer: This product serves as a powerful disinfectant and sanitizer, used in various formulations to kill and inhibit the growth of harmful microorganisms. It is ideal for use in healthcare, sanitation and cleaning products.



Product Name	Pack Size	MRP
Jancitro	1 x 5 ltr	₹ 5500

THE KEY ATTRIBUTES OF JANCITRO 21% ARE AS FOLLOWS:

- **Characteristics/features:**
 - It has the capacity to dissolve bio films and dissolve mineral deposits and hence is highly useful for descaling, decalcification and disinfection of Haemodialysis Machines.
 - This mixed acid containing solution effectively removes scale, cleans organic and inorganic residues and provides broad-spectrum antimicrobial activity.
 - Citric and malic acids effectively dissolve calcium and magnesium deposits (scale) that can build up inside the hemodialysis machine, potentially obstructing flow and affecting the machine's performance.
 - When used at 84° C has not only bactericidal activity but has the capacity to neutralise many viruses.
 - Complies with FDA & GMP standards.
 - It is bio-degradable.
- **Uses/Instructions:**
 - Ensure that the container is properly sealed and at room temperature away from sunlight and always kept in an upright position.
 - It is made up of natural substances and is odourless.
- **Composition:**
 - Citric acid I.P. (21% w/v)
 - Purified water I.P. q.s.
 - Malic acid and Lactic acid added as adjuvants.



Product Name	Pack Size	MRP
Jancitric	1 x 5 ltr	₹ 5500

THE KEY ATTRIBUTES OF JANCITRIC 50% ARE AS FOLLOWS:

- **Characteristics /Features:**
 - Citric acid 50% has excellent capacity to dissolve bio films and dissolve mineral deposits and hence is highly useful for descaling, decalcification and disinfection of Haemodialysis Machines.
 - Suitable NIPRO and B>Brown Machine.
 - Citric acid 50% is highly effective in dissolving calcium and magnesium deposits (scale) that can build up inside the haemodialysis machine, potentially obstructing flow and affecting the machine's performance.
- **Uses/Instructions:**
 - Conforms to International standards and GMP regulations.
 - Used for chemo thermal disinfection of the dialysis machine.
 - It is bio-degradable.
 - Ensure that the container is properly sealed and at room temperature away from sunlight and always kept in an upright position.
- **Composition:**
 - Citric acid anhydride 50%
 - Purified water q.s.



Product Name	Pack Size	MRP
Janis Kidney Plus	1 x 5 ltr	₹ 6500

THE KEY ATTRIBUTES OF KIDNEY PLUS ARE AS FOLLOWS:

- **Characteristics /Features:**
 - Kidney Plus Contains Peracetic acid (PAA), a widely used disinfectant and a top class cold sterilisation agent in haemodialysis, primarily for disinfecting and sterilizing dialyzers, tubing and connectors.
 - Kidney Plus also contains Hydrogen peroxide that is used in haemodialysis for disinfection and sterilization, specifically in the water treatment system (WTS).
 - Kidney Plus also incorporates Acetic acid which is used in haemodialysis, particularly in the acid concentrate, to prevent the precipitation of calcium and magnesium carbonate when mixed with bicarbonate in the dialysis fluid.
- **Uses/Instructions:**
 - Janis Kidney Plus is packed in sealed containers.
 - Janis Kidney Plus should not be used in conjunction with other disinfectants containing aldehydes or halogen derivatives to avoid the possibility of interaction and neutralisation.
 - In case of contact with eyes or skin rinse immediately with cold water and consult a physician.
 - Janis Kidney Plus bears a shelf life of 12 months from the date of manufacture.
 - Keep out of reach of children, away from direct sunlight, in a cool and dark place.
 - Packing : 5 Litres/10 Litres. In a canister with a self vented cap.
- **Composition:**
 - Peracetic acid 4.5%.
 - Hydrogen Peroxide : (48gm of 50%).
 - Acetic acid 25%.
 - Purified water I.P. 10%.



Product Name	Pack Size	MRP
Janis Guard	1 x 1 ltr	₹ 6000
Janis Guard	1 x 5 ltr	₹ 19500

THE KEY ATTRIBUTES OF JANIS GUARD ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Offers 5th generation quaternary ammonium compound which is a powerful wide spectrum biocidal agent that acts on bacteria, fungi, is mildewstatic, virucidal, tuberculocidal and sporicidal.
 - It is eco friendly & user friendly.
 - It is aldehyde and Phenol free.
 - Non corrosive and non hazardous.
- **Uses/Instruction:**
 - For surface disinfection in operation theatre and intensive critical wards, use 0.4% solution (4 ml in 1 litre of cold water).
 - For Instrument disinfection: Soak instruments in 10% (100 ml in 1 litre of water) solution.
 - It is compatible with most materials used in hospitals, industries and laboratories.
 - Securely close the container every time after use and store in a cool and dark place, away from sunlight.
- **Composition:**
 - Didecyl dimethyl ammonium chloride 8.7 %
 - N alkyl dimethyl 8.19%
 - Benzyl ammonium chloride 83.11%



Product Name	Pack Size	MRP
Nebulizers Mini / Prime	1x1	₹ 1799

THE KEY ATTRIBUTES OF NEBULIZER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is used to treat respiratory conditions like asthma, COPD and cystic fibrosis & sometimes used in conjunction with inhalers.
 - Recommended for use in emergency situations to address acute respiratory problems, such as inhalation of toxic substances.
- **Uses/ Instructions:**
 - Nebulizers are commonly used for conditions like asthma, COPD and other lung diseases.
 - Other Lung Diseases: Nebulizers can also be used for conditions like cystic fibrosis, bronchiectasis and other lung diseases where inhaled medications are beneficial.
- **Common Materials Used:**
 - Devic - Type Piston / Compressor nebulizer.
 - Material - ABS plastic (Acrylonitrile Butadiene Styrene) (body) with medical-grade components.
 - Masks Included - Adult mask + Child mask + mouthpiece + air tube + filters.
 - Medication Capacity (chamber) Usually ~ 8 ml.
 - Particle Size (MMAD) ~ 2.0 μ m in cascade-impactor test; may be up to ~ 4.3 μ m depending on other measurement (e.g. DV(50) by Malvern spray etc.).
- **Nebulization Rate** - Not always clearly stated; but the listings mention "controllable nebulization rate" and performance consistent with typical compressors. Might be ~ 0.2-0.3 ml/min or more depending on medicine & settings.
- **Noise Level** - Very low noise with silencer + vibration absorber design. Exact decibel rating not always given; user listings do not always state it.
- **Power Requirement** - Likely AC mains (~ 220-240V, 50 Hz) since design is for home / clinic India.
- **Power Consumption** - 50 Watts.



Product Name	Pack Size	MRP
Digital Thermometer	1x1	₹ 399

THE KEY ATTRIBUTES OF DIGITAL THERMOMETER ARE AS FOLLOWS:

- **Characteristics/Features:**
- It measures the body temperature quickly, offering speed, accuracy and convenience compared to mercury thermometers.
- **Uses / Instructions:**
- To use a digital thermometer, first clean it and switch it on. Choose the method (oral, rectal or armpit) and position the thermometer accordingly.
- **Common Materials Used:**
- Model type - Flexible-tip digital thermometer, white plastic body with flexible tip.
- Battery life - About 7 hours of use.
- Response time - 60 seconds for a reading.
- Color - White.
- Material (exterior) - Plastic; interior material says "mercury" in listing, but given it's "electronic" type, "interior material: mercury" may be a misprint. Usually such digital thermometers have thermistor / electronic sensor, not mercury.
- Insulation material - PVC.
- Usage / type - For temperature measurement; can be used under tongue / armpit / flexible tip etc. (flexible tip helps for comfort).
- Temperature range - 32.0°C to 42.9°C (≈89.6-109.2°F) for body measurement. Some allow a bit lower/higher for ambient or surface use.
- Accuracy - ±0.1°C to ±0.2°C in the normal body range (≈35-42°C).
- Response time - 10-60 seconds, depending on whether oral/underarm or rectal.
- Display - LCD display, possibly with a beep when measurement done, possibly memory of last reading.
- Power Small battery (button cell or AAA) usually; auto-shutoff after some idle period.
- Probe type Flexible tip or rigid tip; washable; bio safe materials.



Product Name	Pack Size	MRP
Steamer	1x1	₹ 1200

THE KEY ATTRIBUTES OF STEAMER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It steams as a natural decongestant for anyone who is suffering from congestion, cold or having trouble with sinusitis.
 - It essentially gives off a fine mist of steam that quickly reaches the inner areas of the nose, sinuses and throat.
 - Widely used to open up blocked nasal passage.
- **Uses / Instructions:**
 - Relieving respiratory symptoms: Steam from a vaporizer can help to loosen mucus, making it easier to cough up and clear congestion in the nasal passages, sinuses and lungs.
 - Soothing irritated airways: The warm, moist steam can soothe irritated nasal passages and throat, providing relief from the symptoms of colds, flu or allergies.
 - Facial steaming: Some vaporizers are designed with facial attachments that can be used for skincare rituals, such as opening pores and improving blood circulation.
 - Enhancing beauty routines: Vaporizers can be used during facials, massaging or clean-up to promote skin smoothness and hydration.
 - Providing comfort: The steam and heat can provide a sense of comfort when feeling unwell, making it easier to relax and sleep.



Product Name	Pack Size	MRP
Hot Water Bottle	1x1	₹ 399

THE KEY ATTRIBUTES OF HOT WATER BAG ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Rubber hot water bottles in hospitals are typically used for their ability to deliver localized heat for pain relief and warmth.
 - Leak proof and easy to clean.
 - Are made from high-quality rubber, designed for better grip and heat distribution.
 - Induces relaxation, relieves stress and eases pains.
- **Uses/ Instructions:**
 - A hot water bag provides a simple method of applying localized heat for pain relief and relaxation. To use one, fill it with warm (not hot or boiling) water, ensuring the bag isn't overfilled and secure the stopper. Wrap the bag in a cloth before applying it to the skin and use for 15–20 minutes at a time.
- **Common Material Used:**
 - Bag Body – Soft Natural Rubber or TPR (Thermoplastic Rubber).
 - Cap – Polypropylene / HDPE.
 - Seal Washer – Rubber/Silicone.
 - Latex – Latex-free (in most variants).



Product Name	Pack Size	MRP
Glucometer with Strips	1x1	Glucometer ₹ 999 Strip (50 PCS) ₹ 1500

THE KEY ATTRIBUTES OF GLUCOMETER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Effective, easy to use, gives accurate reading of blood sugar level.
 - Highly sensitive strips give accurate glucose level readings.
- **Uses/Instructions:**
 - Monitoring Blood Sugar: Glucometers help individuals with diabetes monitor their blood sugar levels, which is crucial for preventing complications and managing the condition. Regular monitoring can help prevent both dangerously low (hypoglycemia) and dangerously high (hyperglycemia) blood sugar levels.
 - Tracking Trends: By regularly testing blood sugar, individuals can identify patterns and trends in their glucose levels, helping them understand how different factors affect their blood sugar.
 - Helps in Adjusting Treatment: Glucometer readings help individuals and their healthcare providers adjust medication dosages, dietary plans and exercise routines to better manage diabetes.
 - Assessing Treatment Effectiveness: Glucometers help evaluate the effectiveness of diabetes therapies, lifestyle changes and other interventions.




Product Name	Pack Size	MRP
Blood Pressure Machine	1x1	₹ 6700

THE KEY ATTRIBUTES OF BLOOD PRESSURE MACHINE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Clinically validated for accurate reading of the blood pressure automatically.
 - Automatic inflation and deflation of the cuff makes the process easier and reduces potential errors.
- **Uses/Instructions:**
 - Diagnosis and Monitoring of Hypertension: BP machines help detect and track high blood pressure, a leading cause of many health problems.
 - Tracking Medication Effectiveness: Patients with hypertension can use BP machines to monitor how well their medications are controlling their blood pressure.
 - Assessing the Effects of Lifestyle Changes: BP machines can help individuals track the impact of lifestyle changes, such as diet and exercise, on their blood pressure.
 - Monitoring for Low Blood Pressure (Hypotension): While less common than high blood pressure, BP machines can also be used to monitor for low blood pressure and its potential causes.
- **Common Materials Used:**
 - Measurement method – Oscillometric (automatic inflation / deflation).
 - Pressure (Blood-Pressure) Range – 0 to ~ 280–300 mmHg.
 - Pulse / Heart-Rate Range ~ 30–200 beats per minute.
 - Accuracy ± 3 mmHg (for pressure), $\pm 5\%$ (for pulse).
 - Resolution 1 mm Hg increments.
 - Cuff Size For upper arm—often ~ 22–42 cm (adult) circumference; some models have small / large cuffs.
 - Display – LCD digital; some with backlight; shows systolic, diastolic, pulse; possibly date/time.
 - Memory Stores – multiple readings; maybe dual user memory; average of readings feature.
 - Power Source – Batteries (often 2–4 \times AA or AAA) or sometimes USB / adapter or rechargeable.
 - Auto Shut-Off – The unit may switch off automatically after few minutes of inactivity.
 - Operating Conditions Ambient temperature – 5–40°C; humidity limits specified (e.g. <85% RH).



Personal Care

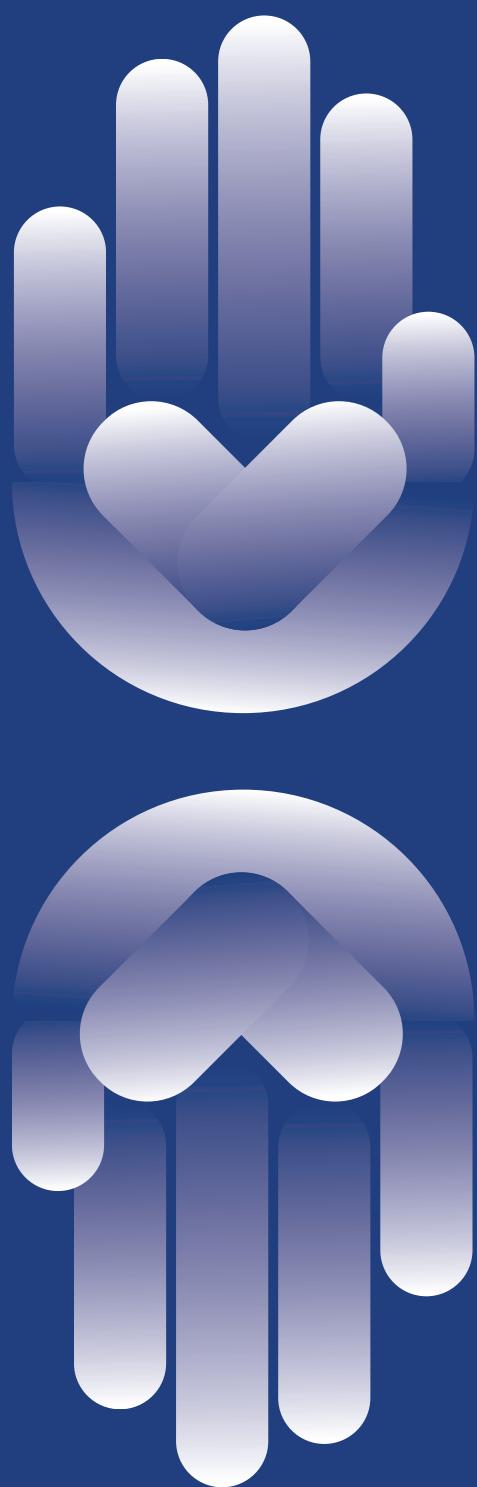



ADULT DIAPERS – Pack Of 5

Sizes	Pack Size	MRP
Adult Diapers - M	1 x 5	₹ 678
Adult Diapers - L	1 x 5	₹ 779
Adult Diapers - XL	1 x 5	₹ 989

ADULT DIAPERS – Pack Of 10

Sizes	Pack Size	MRP
Adult Diapers - M	1 x 10	₹ 1050
Adult Diapers - L	1 x 10	₹ 1160
Adult Diapers - XL	1 x 10	₹ 1265

THE KEY ATTRIBUTES OF ADULT DIAPER ARE AS FOLLOWS:

- **Characteristics /Features:**
 - Specially designed with features such as high absorbency, comfortable fit, odor control, leak protection and breathability.
 - Are comfortable and give the wearer a feeling of physical and psychological well-being.
 - Highly absorbent contents that retain liquid is crucial for maintaining dryness and preventing leakage.
 - Snug fit of the diaper prevents chafing and irritation, allowing for freedom of movement and promoting a more comfortable experience.
- **Uses/Instructions:**
 - Incontinence Management: Adult diapers are a key tool for managing urinary and fecal incontinence, which can occur due to various reasons like aging, medical conditions or injuries.
 - Mobility Issues: People with limited mobility may find it difficult to reach the bathroom in time, making adult diapers a practical option.
 - Neurological Conditions: Individuals with cognitive impairments, such as dementia, may not be able to recognize the need to use the restroom, leading to the need for adult diapers.
- **Common Material Used:**
 - Top Sheet (Inner Layer) – Non-woven, soft-touch hydrophilic material, hypoallergenic.
 - Absorbent Core – Fluff pulp + Super Absorbent Polymer (SAP), multi-layered.
 - Back Sheet (Outer Layer) – Waterproof polyethylene (PE) film or breathable cloth-like film.
 - Leg Cuffs/Elastic – Lycra or elasticized material for snug fit.
 - Acquisition & Distribution Layer (ADL) – Enhances fluid distribution, prevents rewetting.
 - Fasteners – Re-fastenable tape or hook-and-loop type.
 - Waistband (optional) – Elastic waistband for added fit and comfort.



ADULT DIAPERS PANTS / PULL-UPS – Pack Of 5

Sizes	Pack Size	MRP
Adult Diapers – M	1 x 5	₹ 750
Adult Diapers – L	1 x 5	₹ 875
Adult Diapers – XL	1 x 5	₹ 980

ADULT DIAPERS PANTS / PULL-UPS – Pack Of 10

Sizes	Pack Size	MRP
Adult Diapers – M	1 x 10	₹ 1150
Adult Diapers – L	1 x 10	₹ 1200
Adult Diapers – XL	1 x 10	₹ 1295

THE KEY ATTRIBUTES OF ADULT DIAPER - PULL UP PANTS ARE AS FOLLOWS:

- Characteristics/Features:**
 - Specially designed for easy wear and removal, offers leakage protection and made from soft, skin-friendly materials.
 - Absorbs a significant amount of moisture to keep the user dry and comfortable, regardless of the level of incontinence.
 - Bears underwear-like appearance to ensure discretion and minimize the feeling of wearing a diaper.
- Uses/Instructions:**
 - Urinary Incontinence: 1) Helps manage involuntary leakage of urine. 2) Ideal for mild to moderate incontinence, especially for mobile adults.
 - Fecal Incontinence: 1) Provides protection for individuals with bowel control issues. 2) Useful in medical conditions like IBS or after surgeries.
 - Elderly Care: 1) Offers dignity, independence and comfort to seniors. 2) Reduces the need for frequent bed changes and washing.
 - Mobility-Challenged Individuals: 1) Beneficial for people with disabilities or limited mobility. 2) Pull-up design allows for easier wearing and removal.
 - Travel and Long-Duration Activities: 1) Useful during long flights, road trips or situations with limited restroom access. 2) Offers peace of mind for individuals with weak bladder control.
- Common Material Used:**
 - Top Sheet (Inner Lining) – Soft, non-woven fabric; hypoallergenic and breathable.
 - Absorbent Core – Multi-layer fluff pulp with Super Absorbent Polymer (SAP).
 - Back Sheet (Outer Layer) – Cloth-like, breathable, waterproof PE film.
 - Leg Cuffs/Guards – Elasticized cuffs to prevent side leakage.
 - Waistband – Fully elasticated waistband (360° stretch).
 - Acquisition & Distribution Layer (ADL) – Rapid fluid absorption and distribution layer.



Product Name	Pack Size	MRP
Sanitary Pads	1 x 7	₹ 85
Maternity Pads	1 x 10	₹ 399

THE KEY ATTRIBUTES OF SANITARY PADS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Is made from highly absorbent material capable of handling heavy blood flow.
 - Offers leak protection and easy to wear.
 - Is comfortable to wear and maintain good hygiene by preventing odour and irritation.
 - Capable of suppressing odour. Maintains high level of hygiene.
- **Uses/Instructions:**
 - Absorption and Retention: Sanitary pads are primarily designed to absorb and retain menstrual blood and other vaginal fluids.
 - Leakage Prevention: They help prevent leakage by effectively absorbing and containing the flow, ensuring that clothing and surroundings remain clean and dry.
 - Hygiene and Comfort: Sanitary pads contribute to a hygienic and comfortable experience during menstruation by keeping the skin dry and preventing odor.
 - Discreet Carry and Disposal: They are usually individually wrapped for easy carrying and discreet disposal in a trash can or designated disposal box.
- **Common Material Used:**
 - Top Sheet - Soft, non-woven hydrophilic layer (anti-rash, quick-dry, hypoallergenic).
 - Acquisition & Distribution Layer (ADL) - Rapid fluid intake and distribution layer.
 - Absorbent Core - Fluff pulp + Super Absorbent Polymer (SAP); high retention capacity.
 - Back Sheet - Breathable and waterproof PE film or cloth-like material.
 - Wings (if any) - With adhesive for secure placement around underwear.
 - Adhesive - Medical-grade, non-toxic glue for base and wings (if applicable).
 - Wrapper - Individually wrapped in PE wrapper or biodegradable pouch.



Product Name	Pack Size	MRP
Maternity Pads	1 x 10	₹ 399

THE KEY ATTRIBUTES OF MATERNITY PADS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Designed to handle heavy postpartum bleeding (lochia) after childbirth, especially in the first few days.
 - Usually longer, wider, and thicker than regular pads for better coverage and protection. Sizes often range from 320 mm to 450 mm or longer.
 - Gentle, non-woven cottony surface to reduce irritation to sensitive postpartum skin and stitches (especially after vaginal delivery or episiotomy).
 - Free from fragrances, dyes, and chlorine — suitable for sensitive or healing skin
- **Uses/Instructions:**
 - Postpartum Bleeding (Lochia).
 - Primary use is to absorb heavy bleeding after childbirth.
 - Needed regardless of delivery type (vaginal or C-section).
 - Nighttime and Daytime Protection.
 - Wide coverage prevents leaks while lying down or moving around.
 - Helps keep the perineal area clean and dry, preventing infection.
 - Soothing Relief (Optional Use) Can be used as a cold pad (some brands or DIY frozen padsicles) for pain relief and swelling.
- **Common Materials Used:**
 - Top Sheet (Cover) - Non-woven, soft hydrophilic polypropylene (15–20 GSM).
 - Acquisition Layer- Non-woven/perforated PE(20–30 GSM).
 - Absorbent Core- Fluff pulp + Super Absorbent Polymer (SAP) Fluff: 250–400 GSM.
 - SAP: 5–10 g/pad.
 - Back Sheet-Polyethylene film (leak-proof)- 20–30 GSM.
 - Adhesive Layer- Hot melt glue- As required (spot coat).
 - Release Paper- Silicone-coated paper - ~40–60 GSM.



Anesthesia, Respiratory Care & Miscellaneous Products





Product Name	Pack Size	MRP
Janis Neso Mist (Adult)	1x1	₹ 899
Janis Neso Mist (Child)	1x1	₹ 950

THE KEY ATTRIBUTES OF NESO MIST ARE AS FOLLOWS:

- Characteristics/Features:**
 - Designed to match with any Nebulizer machine.
 - Super smooth kink resistant tubing ensures uniform flow rate.
 - Soft, clear aerosol mask, with a snug fit design is ideal for a long term use.
 - Adjustable elastic strip and integrated nose clip for proper positioning of the mask.
 - Plug in the Nebulizer into the standard fitting located at the bottom of the mask.
 - Direct flow design to optimize medicine delivery directly to the patient.
 - Proximal end of the tube connected with funnel shape connectors for easy connection with oxygen source.
- Uses/Instructions:**
 - Set includes Nebulizer mask, Nebulizer Chamber and 200 cms long star lumen tube.
 - Nebulizer rate 3cc in 10 minutes.
 - For single use.
 - Presented as individually packed pouches.



Product Name	Pack Size	MRP
Teflon Nebulizer Mask (Adult)	1x1	₹ 1550
Teflon Nebulizer Mask (Child)	1x1	₹ 1550

THE KEY ATTRIBUTES OF TEFLOL NEBULIZER MASK ARE AS FOLLOWS:

- Characteristics/Features:**
 - A Nebulizer mask helps deliver medication into the lungs for conditions like asthma or COPD.
 - Teflon Nebulizer Mask gives a high aerosol output per unit of time.
 - It gives a high efficiency in absorbing a high percentage of breathable particles out of the total of aerosol emitted especially in patients with asthma and COPD commonly use inhaled drugs.
 - The Teflon nebulizer consist of five main parts, A Compressor (the compressor is the part that fuels the working of the nebulizer system), Tubing, Nebulizing chamber, Face mask/Mouthpiece, Nebulizer filter.
 - It works by using air compressor to create the mist containing the medicine.
- Uses/Instructions:**
 - A Nebulizer mask covers the nose and mouth for medication delivery through both airways. A mouthpiece directs medication through the mouth. These are designed for individuals who find it difficult to hold the device easily including children or the elderly.



Product Name	Pack Size	MRP
Janis Oxee Mask (Adult)	1x1	₹ 475
Janis Oxee Mask (Child)	1x1	₹ 410

THE KEY ATTRIBUTES OF OXEE MASK ARE AS FOLLOWS:

- Characteristics/Features:**
 - Comfortable, secure and effective at delivering oxygen.
 - Incorporates kink resistant tubing to ensure uniform flow.
 - Made from special grade non toxic PVC.
 - Provided with 2 holes for escape of the carbon dioxide exhaled by the patient.
 - Adjustable elastic strip and integrated nose clip for proper positioning of the mask.
 - Proximal end of the tube connected with a funnel shaped connector with oxygen source.
 - Star lumen main tube to avoid accidental blockage.
 - Sterile and individually packed.
- Uses/ Instructions:**
 - Oxygen Delivery:** The primary use of an oxygen mask is to deliver supplemental oxygen to individuals who are not getting enough oxygen from the air they are breathing.
 - Medical Applications:** Oxygen masks are commonly used in hospitals, ambulances and other healthcare settings to treat various respiratory conditions.
 - Hypoxia:** In situations where the blood oxygen level is low (hypoxia), an oxygen mask helps increase oxygen saturation in the blood.
 - COPD and Asthma:** Patients with COPD or asthma may use oxygen masks to alleviate shortness of breath and improve oxygen levels during exacerbations.
 - Trauma and Emergency Situations:** Oxygen masks are crucial in emergency situations, such as after trauma or smoke inhalation, to provide essential oxygen support.
 - High-Flow Oxygen Therapy:** Some oxygen masks are designed for high-flow oxygen therapy, providing a higher concentration of oxygen to patients with severe respiratory distress.
 - Carbon Monoxide Poisoning:** Oxygen masks are vital in cases of carbon monoxide poisoning, as they help to displace carbon monoxide in the blood with oxygen.
 - Decompression Sickness:** In some cases, oxygen masks are used in hyperbaric chambers to treat decompression sickness.



Product Name	Pack Size	MRP
Teflon Oxygen Mask (Adult)	1x1	₹ 1550
Teflon Oxygen Mask (Child)	1x1	₹ 1550

THE KEY ATTRIBUTES OF TEFLON OXYGEN MASK ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Teflon oxygen masks, also known as PTFE (Polytetrafluoroethylene) oxygen masks, are made of a material known for its chemical resistance, low friction and non-stick properties. It is highly resistant to most chemicals, including strong acids, solvents and bases, making it ideal for environments that might be corrosive to other materials.
 - As the mask is made up of Teflon material it is resistant to adhesion.
 - The mask makes the flow of oxygen, consistent and on wearing the patient feels Comfortable, as it does not cause rashes on the cheek and neither mist in the mask.
- **Uses/Instructions:**
 - Teflon oxygen masks are used to deliver supplemental oxygen to patients in need. These masks are primarily used in hospitals and for home oxygen therapy for delivering supplemental oxygen to individuals with respiratory conditions or those experiencing low blood oxygen levels.
 - The mask is typically attached to an oxygen supply system, either a flow meter or a humidifier.
 - The mask is placed over the patient's nose and mouth, ensuring a snug but comfortable fit.
 - It should be kept clean and free from debris or damage to ensure proper functioning.





Products Name	Pack Size	MRP
Janis Oxee Set (Adult)	1x1	₹ 540
Janis Oxee Set (Child)	1x1	₹ 440
Janis Oxee Set (Neo)	1x1	₹ 410

THE KEY ATTRIBUTES OF OXEE SET ARE AS FOLLOWS:

- **Characteristics/Features:**
- Is comfortable, easy to use, lightweight and disposable, allowing for unimpeded communication and minimal irritation. Super smooth kink resistant tubing ensures uniform flow rate delivering the necessary oxygen concentration while being easily tolerated by the patient.
- Cannula made from non-toxic, medical-grade materials and is crush-resistant.
- Oxygen can be administered at the rate of 6 litres/ minute or less. Flow rate of 4 litres per minute or less do not need to be humidified. Sterile and individually packed.

- **Uses/ Instructions:**
- Used to deliver supplemental oxygen. It consists of two prongs that are inserted into the nostrils, connected to a tubing that leads to an oxygen source.
- The primary use is to increase the oxygen concentration in the air inhaled by a patient, helping them breathe more easily in conditions like COPD, pneumonia or other respiratory issues.
- Instruction: Wash hands thoroughly before handling the cannula. Check the oxygen source (tank or concentrator) and ensure it's properly connected and functioning. Inspect the nasal cannula prongs and tubing for any damage or blockage. Insert the two prongs of the nasal cannula into your nostrils, ensuring they are positioned comfortably and securely. Carefully wrap the tubing around your ears, securing them comfortably. Adjust the slider or elastic band on the tubing to find a snug but comfortable fit. Adjust the flow rate on the oxygen source as prescribed by your healthcare provider. Ensure there is gas flow from the nasal prongs by checking the flow meter.



Products Name	Pack Size	MRP
HI Mask (Adult)	1x1	₹ 650
HI Mask (Child)	1x1	₹ 650

THE KEY ATTRIBUTES OF HI MASK ARE AS FOLLOWS:


- Characteristics/Features:**
 - Designed to deliver a high concentration of Oxygen (60 – 80%) to the patients.
 - Has a one way valve system that prevents exhaled oxygen from mixing with oxygen from the reservoir bag. Provided with a reservoir bag that holds a large volume of oxygen, ensuring a sufficient supply to meet the patient's breathing needs, even with variations in breathing patterns.
- Uses/ Instructions:**
 - Used to deliver high levels of oxygen to patients, with respiratory distress or needing increased oxygen saturation such as asthma attacks, pneumonia, hypoxia or COPD exacerbations. Designed to minimize the amount of room air the patient inhales, allowing for a higher concentration of oxygen to be delivered.
 - Instruction: Attach to oxygen source: Connect the mask's tubing to the oxygen flow meter and ensure a secure fit. Open the safety valve and allow the reservoir bag to fill with oxygen. Set the flow rate on the oxygen flow meter, typically between 10 and 15 litres per minute, to keep the reservoir bag partially deflated during inhalation. Fit the mask snugly over the patient's nose and mouth, ensuring a tight seal. Adjust the elastic straps to secure the mask comfortably and prevent leaks.



Product Name	Pack Size	MRP
Venturi Mask (Adult)	1x1	₹ 450
Venturi Mask (Pediatric)	1x1	₹ 450

THE KEY ATTRIBUTES OF VENTURI MASK ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Offers precise oxygen concentration control, a comfortable secure fit, easy cleaning and sterilization.
 - Designed to deliver accurate and specific quantity of oxygen to the patients with respiratory conditions.
 - It is a transparent mask with multiple colour coded valves or adapters which allow adjustment of the concentration of oxygen administered.
- **Uses/ Instructions:**
 - Controlled oxygen therapy: Venturi masks are designed to provide a specific oxygen concentration, allowing for precise adjustment of oxygen delivery based on individual patient needs.
 - COPD management: They are particularly useful for patients with COPD (Chronic Obstructive Pulmonary Disease) to prevent hypercarbia (excess carbon dioxide in the blood) by delivering controlled oxygen concentrations.
 - Acute respiratory failure: Venturi masks are used to treat acute respiratory failure in patients with COPD, ensuring adequate oxygen delivery.
 - High-flow ventilation: They can be used for high-flow ventilation, especially during ophthalmic surgery under local anesthesia.
 - Preoxygenation: Venturi masks are used for preoxygenation before various procedures, such as surgeries, ensuring adequate oxygen levels before the procedure.
 - Humidification and oxygen therapy: They can be used with air-entrainment nebulizers to provide both humidification and oxygen therapy.

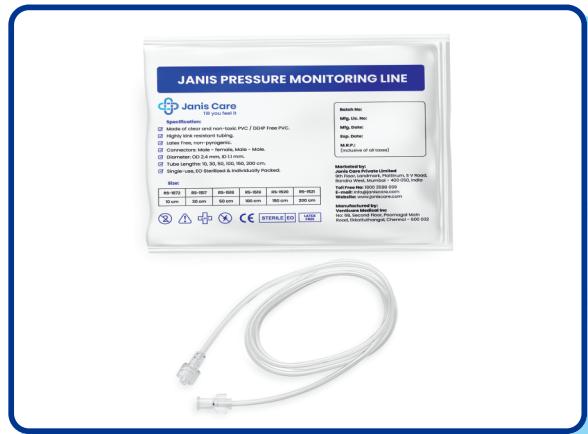

Products Name	Pack Size	MRP
PMO Line - 100 cm	1x1	₹ 300
PMO Line - 150 cm	1x1	₹ 300
PMO Line - 200 cm	1x1	₹ 300

THE KEY ATTRIBUTES OF PMO LINE ARE AS FOLLOWS:

- Characteristics/Features:**
 - Is made from a clear, non toxic PVC –medical grade, has a small bore and is able to withstand high pressures.
 - Has secure Luer-lock connectors and is suitable for connecting syringe infusion pumps and patients.
 - Small bore design for high pressure applications.
 - Sterile and individually packaged for single use.
 - Designed to be kink-resistant to maintain flow.
 - Available in various lengths to suit different clinical needs.
- Uses/Instructions:**
 - Primarily used in cardiac surgery for invasive blood pressure monitoring and connecting syringe infusion pump and the patient.



Products Name	Pack Size	MRP
Vein - O - Line - 10 cm	1x1	₹ 298
Vein - O - Line - 100 cm	1x1	₹ 397
Vein - O - Line - 150 cm	1x1	₹ 450
Vein - O - Line - 200 cm	1x1	₹ 480

THE KEY ATTRIBUTES OF VEIN O LINE ARE AS FOLLOWS:

- Characteristics/Features:**
 - Is a medical device used to extend the length of the existing intravenous line.
 - Used primarily to connect transducer system to the indwelling catheter.
 - Connects patients I.V. Catheter to the I.V infusion or medicine administration set.
 - Sterile and disposable. Ensures patient safety and prevents cross-contamination.
 - Sterile and individually packed.
- Uses/Instructions:**
 - "Vein O Line" refers to a medical device, specifically a catheter or extension line, used for intravenous (IV) procedures.
 - It's a sterile, disposable and flexible tube designed for connecting to IV equipment and accessing veins for various purposes like medication delivery or blood draws.
 - Allows its use in MRI procedures without interference.
 - Available in various lengths (10 cm, 50 cm, 150 cm, 200 cm) to suit different clinical needs.
 - Used for administering IV fluids, medications, blood transfusions, blood sampling.

Abdominal Drainage Kit (ADK)



Product Name	Pack Size	MRP
Abdominal Drainage Kit - 16	1x1	₹ 525
Abdominal Drainage Kit - 20	1x1	₹ 525
Abdominal Drainage Kit - 24	1x1	₹ 525
Abdominal Drainage Kit - 28	1x1	₹ 525
Abdominal Drainage Kit - 32	1x1	₹ 525

THE KEY ATTRIBUTES OF ABDOMINAL DRAINAGE KIT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - An abdominal drainage kit is a sterile, single-use medical device used to drain fluid from the abdominal cavity, typically after surgery or in cases of fluid buildup.
 - It includes a drainage catheter with a collection bag, designed for efficient drainage of fluids and easy visualization.
 - The catheter and tubing are made of a non-toxic, medical-grade polyvinyl chloride (PVC).
 - A radio-opaque line is typically present along the catheter's length, enabling easy visualization via X-ray.
 - A bag is attached to the catheter to collect the drained fluid. The collection bag often has a graduated scale for easy volume measurement with a handle for easy carrying and hanging.
- **Uses/Instructions:**
 - The catheter is a flexible tube designed to be inserted into the abdominal cavity to drain fluid.
 - Sterile and Non-Pyrogenic.
 - Used to drain excess fluid that has built up in the abdominal cavity due to conditions like ascites (fluid in the peritoneal cavity).



Product Name	Pack Size	MRP
Janis Suction Set	1x1	₹ 1050

THE KEY ATTRIBUTES OF SUCTION SET ARE AS FOLLOWS:

- Characteristics/Features:**
 - A Yankauer suction set is a medical device with a rigid angled suction tip connected to a flexible, ribbed tube used to remove fluids and secretions, post surgical procedure or to clear the airway.
 - Set is provided with a Yankauer suction tip mounted on 2 meters long ribbed tube provided with universal connector at both the ends.
 - A rigid, angled suction tip with a large opening and multiple side holes for effective fluid and debris removal.
 - Universal connectors are moulded from soft PVC to accommodate suction tip at one end.
- Uses/Instructions:**
 - A Yankauer suction set is a medical device used to remove fluids and secretions, commonly during or after surgical procedures or to clear the airway.
 - Used to suction secretions and prevent aspiration in airway management.



Products Name	Pack Size	MRP
Reusable Ambu Bag (Adult)	1x1	₹ 5500
Reusable Ambu Bag (Child)	1x1	₹ 5250
Reusable Ambu Bag (Neo)	1x1	₹ 5100

THE KEY ATTRIBUTES OF REUSABLE AMBU BAGS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Ambu bags, also known as Bag Valve Mask (BVM) resuscitators, are characterized by their ability to withstand repeated use and sterilization.
 - The device consists of a mask and a cylinder attached to it.
 - Can withstand multiple autoclaving cycles, allowing for thorough sterilization.
 - A self inflating bag is used to deliver air (breath) manually (in order to maintain adequate oxygen supply to the brain and other vital organs) when patient can not breath by himself.
- **Uses/Instructions:**
 - Highly useful for pulmonary resuscitation for breathless patients.
 - Offer a durable and reliable solution for providing artificial ventilation and oxygenation in various medical settings.



Product Name	Pack Size	MRP
Three Balls Spirometer	1x1	₹ 1075

THE KEY ATTRIBUTES OF THREE BALLS SPIROMETER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a breathing device used to encourage deep breaths and improve lung function.
 - It has three chambers, each with a ball that rises as the patient inhales.
 - Three color-coded balls indicate different flow rates (1200cc/sec, 900cc/sec, 600cc/sec) helping the patient visualize their breathing effort and progress. The rising balls provide a visual cue for patients to understand their inhalation effort and progress
 - Has a flexible tube holder.
 - Non sterile, individually packed in a pouch.
- **Uses/Instructions:**
 - Sit upright in a comfortable position, such as on the edge of a bed or in a chair. Hold the spirometer upright, at eye level. Insert the mouthpiece into your mouth and create a tight seal with your lips. Breathe in (inhale) slowly through the mouthpiece as slowly and deeply as you can, allowing the balls to rise.
 - It works by providing visual feedback as you inhale, with the balls rising to show the depth of your breath. The goal is to lift the balls as high as possible, while maintaining a slow and controlled inhalation
 - Regular use can help improve lung capacity and reduce the risk of complications like pneumonia and atelectasis.
 - It encourages deep, slow breathing, which can relieve anxiety, reduce shortness of breath and strengthen the respiratory system.
 - Used for patients with conditions like TB, pneumonia, COPD and asthma.



Product Name	Pack Size	MRP
Nibp Cuff (Adult)	1x1	₹ 550
Nibp Cuff (Child)	1x1	₹ 550

THE KEY ATTRIBUTES OF NIBP CUFF ARE AS FOLLOWS:

- **Characteristics/Features:**
 - NIBP (non-invasive blood pressure) cuffs are designed for accurate and comfortable blood pressure measurement.
 - It has features like adjusting arrangement for individual patients fit and are durable, made of easy-to-clean materials.
 - Made from soft, moisture-repellent materials, soft rounded edges and easy-to-handle connectors.
- **Uses/Instructions:**
 - A NIBP (non-invasive blood pressure) cuff is an inflatable band used to measure blood pressure.
 - Place about 1 inch above the elbow on the patient's arm, Align the arrow on the cuff with the brachial artery on the patient's arm.
 - Tighten the cuff evenly.
 - Inflate the cuff to a pressure slightly above the expected systolic pressure.
 - Slowly release the pressure while listening for the sounds indicating systolic and diastolic pressures and then deflat to determine blood pressure readings. The device will display the systolic and diastolic blood pressure readings.

Disposable Monopolar Pencils 3 -pin



Products Name	Pack Size	MRP
Disposable Monopolar Pencils 3-pin	1x1	₹ 1100

THE KEY ATTRIBUTES OF DISPOSABLE MONOPOLAR PENCILS 3-PIN ARE AS FOLLOWS:

- Characteristics/Features:**
 - It is a disposable monopolar pencil with a 3-pin connector, a medical device used in electrosurgery, primarily for cutting and coagulating biological tissue.
 - It has 2 push buttons/ESU pencil.
 - Is disposable. Reduces the risk of cross-contamination and eliminates the need for sterilization.
 - It employs radio frequency alternating current (RFAC) to generate heat, which is then focused at the tip of the pencil to achieve the desired effect.
- Uses/Instructions:**
 - It is used in surgery to cut and coagulate tissue, controlling bleeding and allowing for precise dissection.
 - The heated tip of the pencil cuts through tissue, allowing surgeons to make incisions and create access points for surgical procedures.
 - By applying heat to blood vessels, the pencil can stop bleeding and seal off damaged vessels, minimizing blood loss during surgery.
 - In some cases, the pencil can be used to destroy tissue or vaporize it, a process known as desiccation or fulguration.



Product Name	Pack Size	MRP
Feedy (5)	1x1	₹ 68
Feedy (6)	1x1	₹ 68
Feedy (7)	1x1	₹ 68
Feedy (8)	1x1	₹ 68
Feedy (9)	1x1	₹ 68
Feedy (10)	1x1	₹ 68

THE KEY ATTRIBUTES OF INFANT FEEDING TUBE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Non-toxic PVC material, designed for gentle and accurate delivery of nourishment.
 - It has a coned distal end for smooth intubation, two lateral eyes for efficient drainage and administration and a proximal end with a female Luer mount for easy connection to feeding funnels or syringes.
 - It is marked with measurement markings at every cm, starting from the tip at 5cm to 40cm, to ensure accurate placement.
 - Provides a low-friction, non-traumatic intubation experience.
 - Enhances visibility during X-ray or other imaging techniques, aiding in proper placement confirmation.
- **Uses/Instructions:**
 - It is used to deliver nutrition to babies who cannot eat or drink by mouth, such as those who are premature, have feeding difficulties or have certain medical condition.
 - It is inserted through the nose or mouth and into the stomach or sometimes the small intestine, to provide milk, formula or medications.



Product Name	Pack Size	MRP
Ryles Tube (6)	1x1	₹ 85
Ryles Tube (8)	1x1	₹ 85
Ryles Tube (10)	1x1	₹ 85
Ryles Tube (12)	1x1	₹ 85
Ryles Tube (14)	1x1	₹ 85
Ryles Tube (16)	1x1	₹ 85
Ryles Tube (18)	1x1	₹ 85
Ryles Tube (20)	1x1	₹ 85

THE KEY ATTRIBUTES OF RYLES TUBE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a disposable polyvinyl tube, also known as a nasogastric tube (NG tube), is a soft, flexible tube used for feeding, administering medication or removing stomach contents.
 - It has a smooth, low-friction surface to facilitate insertion and a conal distal end with radiopaque markings for visualisation, multiple lateral eyes for efficient drainage and aspiration.
 - It is made of non-toxic, medical-grade PVC with a silicone elastomer coating to minimize adverse effects.
 - A radiopaque line is present throughout the tube to aid in confirming proper placement via X-ray.
 - Openings on the side of the tube allow for efficient drainage and aspiration of stomach contents.
- **Uses/Instructions:**
 - Used for managing patients who cannot swallow or eat normally or for various medical procedures.
 - Especially in patients who cannot eat or swallow, a Ryle's tube can deliver liquid nutritional formulas directly into the stomach.
 - Medications can be given through the tube, bypassing the need for oral or intravenous routes.
 - It can be used to remove stomach contents, which is helpful in cases of intestinal obstruction, poisoning or after surgery.
 - By removing gas and fluids, the tube can relieve symptoms of stomach distension and discomfort, particularly in cases of intestinal obstruction or ileus.
 - Stomach contents can be aspirated and analyzed to aid in diagnosing various conditions.
 - The tube can be used to empty the stomach before surgery or to prevent vomiting after surgery.
 - Ryle's tubes can also be used for gastric lavage (washing out the stomach in cases of poisoning), or for administering radiographic contrast during imaging studies.



Product Name	Pack Size	MRP
Endotracheal Tube Plain	1x1	₹ 465

THE KEY ATTRIBUTES OF ENDOTRACHEAL TUBE PLAIN ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Made from non-toxic, transparent, thermo-sensitive PVC.
 - Disposable and sterile for single use.
 - Standard 15mm connector for ventilation circuits.
 - Graduated markings for insertion depth and internal/external diameter indication.
 - Smooth, beveled tip for easy insertion.
 - Murphy's Eye: A small opening near the tip to reduce the risk of the tube being blocked by the tracheal wall.
 - A line that allows for X-ray verification of proper placement.
 - Designed to be kink-resistant to prevent airflow blockage.
- **Uses/Instructions:**
 - Provides an alternative pathway for airflow when the natural airway is blocked. It is a plain (uncuffed) endotracheal tube a medical device used to establish and maintain an airway during procedures like surgery or for respiratory support.
 - Used to establish and maintain a direct airway for breathing, especially when a patient needs mechanical ventilation or support in managing their respiratory system.
 - Provides a direct and unobstructed pathway for air to enter and exit the lungs.
 - Allows for the delivery of oxygen and inhaled gases to the lungs via a ventilator.
 - The tube's radiopaque line aids in confirming its correct position within the trachea.



Product Name	Pack Size	MRP
Endotracheal Tube Cuffed	1x1	₹ 875

THE KEY ATTRIBUTES OF ENDOTRACHEAL TUBE CUFFED ARE AS FOLLOWS:

- **Characteristics/Features:**
- Cuffed endotracheal tubes have an inflatable balloon, called a cuff, at the distal end that seals the trachea, preventing air and fluid from leaking around the tube. The cuff is inflated via a pilot balloon, and its pressure monitored to ensure proper sealing without causing damage to the tracheal wall. The cuff is inflated using a syringe attached to the pilot balloon, creating a seal against the tracheal wall.
- The pilot balloon is a smaller inflatable balloon connected to the cuff via a hollow catheter. It serves as a tactile gauge for cuff pressure and helps maintain a stable cuff pressure. Designed to ensure quick and easy cuff inflation and deflation.
- The seal provided by the cuff allows for effective delivery of oxygen and anesthesia gases through the tube.
- **Uses/Instructions:**
- Monitoring cuff pressure is essential to ensure a proper seal without causing tracheal injury. A pressure manometer can be used to measure cuff pressure and the recommended range is typically 20–30 cm H₂O.
- It is used to create a seal in the airway, ensuring effective ventilation and preventing aspiration of stomach contents and other secretions. The inflated cuff prevents air leaks around the tube, ensuring that all gas flow goes directly into the lungs. Cuffed tubes are commonly used during surgeries, particularly those involving the upper abdomen or chest, to provide reliable and controlled ventilation.
- Used in lung disease, chest trauma or airway obstruction.





Product Name	Pack Size	MRP
Guedel Airway	1x1	₹ 165

THE KEY ATTRIBUTES OF ORO-PHARYNGEAL AIRWAY ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a curved, rigid device inserted into the mouth to prevent the tongue from blocking the airway in unconscious patients. It has a flange to sit outside the mouth, a bite block and a central channel for airflow.
 - Made of hard plastic or rubber, it is curved to conform to the shape of the oropharynx.
 - A flat, external part that rests on the patient's lips, preventing the airway from entering too far into the mouth.
 - Incorporates a hardened area designed to protect the teeth from damage.
 - A hollow passage that allows air to flow through the device.
 - The end of the device that sits at the base of the tongue.
- **Uses/Instructions:**
 - The primary function of an OPA is to prevent obstruction of the airway by the tongue, which can occur when a person is unconscious and their jaw muscles relax.
 - To keep the airway open by lifting the tongue away from the posterior pharyngeal wall.
 - Used in unconscious patients with no gag reflex, preventing the tongue from obstructing the airway.



Product Name	Pack Size	MRP
Suction Catheter (6)	1x1	₹ 88
Suction Catheter (8)	1x1	₹ 88
Suction Catheter (10)	1x1	₹ 88
Suction Catheter (12)	1x1	₹ 88
Suction Catheter (14)	1x1	₹ 88
Suction Catheter (16)	1x1	₹ 88
Suction Catheter (18)	1x1	₹ 88

THE KEY ATTRIBUTES OF SUCTION CATHETER PLAIN ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Suction catheter is a medical device used to remove secretions from a patient's airway.
 - Contains a smooth, flexible tube with a rounded,atraumatic tip and a connector for attaching to a suction device.
 - It is non-toxic, non-irritant medical-grade PVC material, smooth tubing for easy insertion, a distal end with one or more lateral eyes for suctioning and color-coded connectors for size identification.
- **Uses/Instructions:**
 - It is used to remove respiratory secretions from a patient's airway, specifically the mouth, trachea and bronchial tubes.
 - It a disposable medical device designed to clear obstructions like mucus buildup, thereby improving breathing and preventing potential respiratory issues.
 - By removing obstructions, the catheter helps ensure the airway remains open and unobstructed, allowing for proper gas exchange.





Product Name	Pack Size	MRP
Foley Catheter (Adult)	1x1	₹ 387
Foley Catheter (Pediatric)	1x1	₹ 387

THE KEY ATTRIBUTES OF FOLEY CATHETER - 2WAY ARE AS FOLLOWS:

- **Characteristics/Features:**
 - A 2-way Foley catheter is a type of indwelling urinary catheter with two distinct channels (lumens). One lumen is for draining urine and the other is for inflating a balloon that anchors the catheter in the bladder.
 - As the name suggests, a 2-way Foley catheter has two separate channels running along its length.
 - One lumen is open at both ends and allows urine to flow from the bladder into a drainage bag.
 - The second lumen connects to a balloon located at the catheter's tip. This balloon is inflated with sterile water or saline after insertion, anchoring the catheter within the bladder and preventing it from slipping out.
 - Foley catheters are measured in French (Fr) units, where 1Fr equals 0.33 mm in diameter. They range in size from 8Fr to 36Fr.
 - Sterile, for single use.
- **Uses/Instructions:**
 - Used to drain urine, especially in situations where patients are unable to void on their own or need urinary output monitored.
 - To drain the bladder following an episode of urine retention and also following urinary tract or bladder surgery.



Product Name	Pack Size	MRP
Silicon Foley Catheter (Adult)	1x1	₹ 355
Silicon Foley Catheter (Pediatric)	1x1	₹ 355

THE KEY ATTRIBUTES OF SILICON FOLEY CATHETER - 2 WAY ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is an indwelling urinary catheters made of 100% medical-grade silicone.
 - Offers several advantages over traditional latex catheters, including improved biocompatibility, reduced encrustation and blockage rates and are ideal for patients with latex allergies.
 - The "2-way" refers to the two lumens or channels within the catheter: one for drainage and one for inflation of the balloon.
 - The smooth, tapered tip facilitates easier insertion into the urethra.
 - The balloon expands symmetrically, ensuring proper retention within the bladder.
- **Uses/Instructions:**
 - The catheter is inserted into the urethra and positioned within the bladder to drain urine.
 - Used after surgeries to drain urine and prevent bladder distension.
 - Can be used for extended periods in patients needing continuous drainage.
 - Can be used to deliver medication directly into the bladder, such as during chemotherapy for bladder cancer.



Product Name	Pack Size	MRP
Urine Collection Bag (Uro Bag)	1x1	₹ 390

THE KEY ATTRIBUTES OF URINE COLLECTION BAG ARE AS FOLLOWS:

- **Characteristics/Features:**
- Hermetic closure to prevent leakage and unpleasant odours. Anti-reflux valve, which prevents urine from flowing back into the catheter or tube, reducing the risk of infection.
- **Uses/Instructions:**
- Urine collection bags are used to collect and store urine, typically from individuals who cannot easily or safely urinate on their own.
- Used in medical conditions like incontinence, urinary retention or surgeries involving the urinary tract.
- Also used for monitoring urine output in various clinical settings.



Product Name	Pack Size	MRP
Urine Bag with Measured Volume Chamber	1x1	₹ 588


THE KEY ATTRIBUTES OF URINE BAG WITH MEASURED VOLUME CHAMBER ADULT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - An adult urine bag with a measured volume chamber is a medical device used to collect and measure urine output, often used in hospitals or home care for patients with urinary incontinence, urinary retention or post-surgery needs.
 - The bag has a clear, transparent chamber with graduations, allowing easy and accurate readings.
 - Has a kink resistant tube for easy drainage.
 - A wide outlet allows for easy and spillage-free emptying of the urine bag.
- **Uses/Instructions:**
 - Has a clear chamber, marked in milliliters, enables precise readings of urine output.
 - Used in hospitals, ICUs and for home care, especially for patients with urinary incontinence, urinary retention or those recovering from surgery.



Orthopedic Products



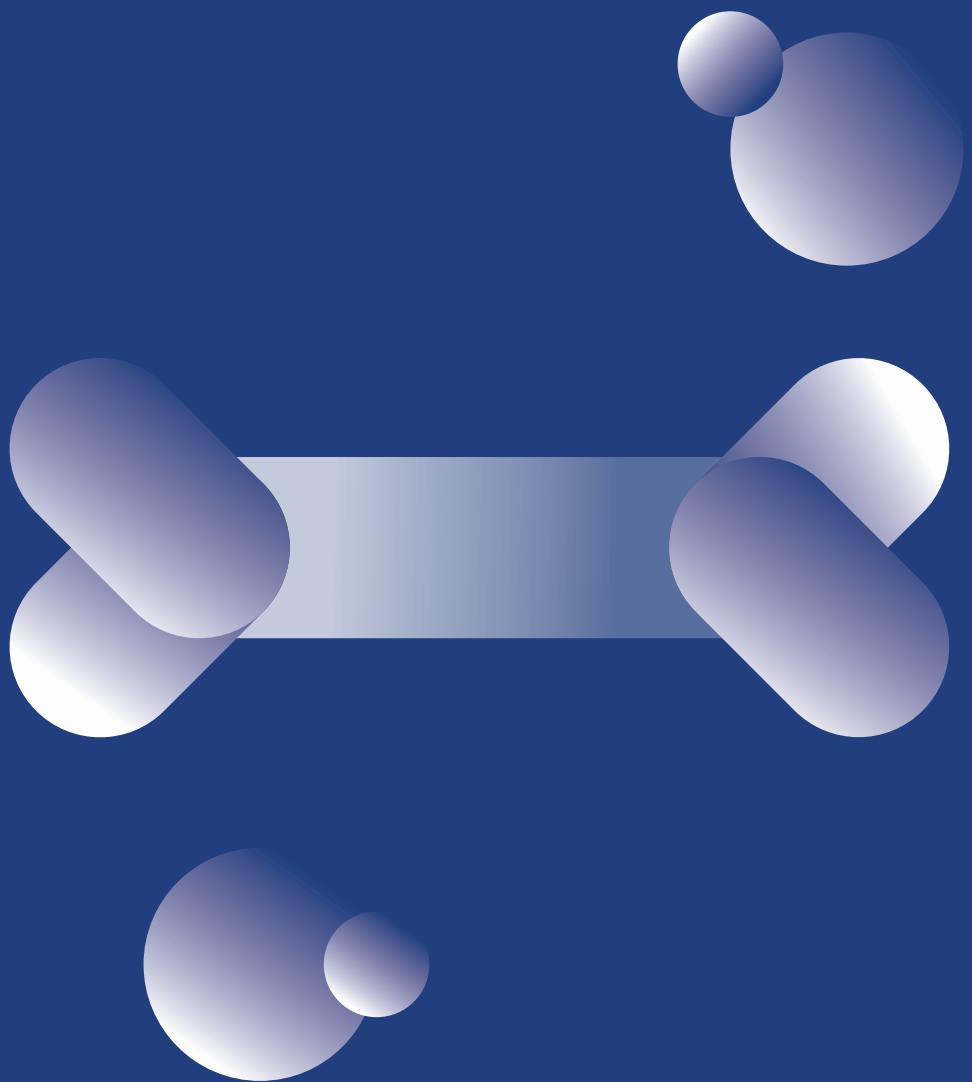



Product Name	Pack Size	MRP
Abdominal Belt	1x1	₹ 975

THE KEY ATTRIBUTES OF ABDOMINAL BELT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a supportive garment designed to encircle the abdomen, offering varying degrees of compression and support.
 - It is made up of elastic and breathable materials, adjustable straps and wide coverage.
- **Uses/Instructions:**
 - For use post-surgery recovery, pregnancy support and managing conditions like low back pain.
 - Provide compression and support to the abdominal muscles after surgeries, especially c-sections to promote healing and reduce swelling.
 - Can be used for speeding up recovery post abdominal surgery.
 - After childbirth, helps support the abdominal muscles, uterus and lower back, aiding in postpartum recovery and potentially reducing belly size.
 - Relieves pain in the abdominal area, back and lower back by providing support and compression.
 - Helps individuals sit straighter, leading to improved posture and reduced back and neck pain.



Product Name	Pack Size	MRP
Arm Sling	1x1	₹ 575

THE KEY ATTRIBUTES OF ARM SLING ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a medical support device used to immobilize and support the arm, shoulder and sometimes the wrist during recovery from injuries or surgeries.
 - Consists of a pouch or cradle that holds the arm in place and a strap system to secure it.
 - Has an adjustable strap, breathable material and a comfortable design.
- **Uses/Instructions:**
 - Primary function is to stabilize the arm, preventing unnecessary movement and promoting healing.
 - Primarily used to support and immobilize an injured arm, shoulder or elbow.
 - Helps prevent further movement and promotes healing after injuries like fractures, dislocations, sprains or following surgery.
 - Provides external support, minimizing the risk of further injury or damage to the healing area.



Product Name	Pack Size	MRP
Cervical Collar	1x1	₹ 525

THE KEY ATTRIBUTES OF CERVICAL COLLAR ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Ergonomically designed to support neck and spine.
 - It is a simple-to-use, noninvasive, nonintrusive, cheap and comfortable external support that can be used after cervical surgery or injuries.
 - It limits cervical and head movements such as antero-posterior flexion/extension and backward motion of the mandible.
- **Uses/Instructions:**
 - Cervical collars, also known as neck braces or C collars, are used to support your spinal cord and head.
 - Commonly used for treatment for neck injuries, neck surgeries and some instances of neck pain.



Product Name	Pack Size	MRP
Knee Caps	1 x 1	₹ 475

THE KEY ATTRIBUTES OF KNEE CAP ARE AS FOLLOWS:

- **Characteristics/Features:**
- Designed to fit the knees snugly as an external support and cover made up of a stretchable fibre which support the knee with a certain amount of pressure and compression.
- **Uses/Instructions:**
- Protects knees and promotes the healing process preventing further injury to the knee joint.
- Provides relief from knee pain.



Product Name	Pack Size	MRP
Lumbo Sacral Belt	1x1	₹ 475

THE KEY ATTRIBUTES OF LUMBO SACRAL BELT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a supportive garment worn around the lower back and abdomen to provide stability and reduce pain.
 - Designed to offer support for the lumbar spine, which is crucial for managing conditions like back strains, sprains, sciatica, muscle spasms and pain from herniated discs.
 - Offers adjustable straps or a double-pull mechanism, allowing for customization of compression and fit.
- **Uses/Instructions:**
 - Provide support to the lower back and abdomen, helping to stabilize the lumbar spine.
 - By minimizing excessive movement, the belt can help reduce pain and discomfort associated with various back conditions.
 - Made from durable, elastic or neoprene materials that offer flexibility and support.
 - Can be worn during daily activities, including walking, sitting and lifting, offering support and pain relief.



Product Name	Pack Size	MRP
Rib Belt	1x1	₹ 795

THE KEY ATTRIBUTES OF RIB BELT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - A rib belt is a medical device that helps manage pain and promote healing after rib injuries, fractures or post-surgical procedures.
 - It is an elastic strap designed to support and stabilize the chest area, minimizing movement and reducing discomfort.
 - Made from breathable, stretchable and soft elastic fabric to ensure comfort and proper fit.
 - Adjustable Velcro or hook-and-loop closures, allowing for a customized fit and controlled compression.
- **Uses/Instructions:**
 - The gentle compression provided by the belt helps reduce pain associated with ribcage movement, allowing for more comfortable healing.
 - Used to provide support and compression to the rib cage, aiding in recovery from injuries like rib fractures, sprains and dislocations, as well as post-surgical recovery.
 - Helps limit chest expansion, reduce pain and promote healing by stabilizing the area and protecting surgical incisions during coughing bouts.
 - After a rib fracture, sprain or other injury, a rib belt can help stabilize the injured area and reduce pain associated with breathing and movement.
 - The compression provided by the belt can help alleviate pain and discomfort caused by rib injuries or post-surgical conditions.



Product Name	Pack Size	MRP
Universal Shoulder Immobilizer	1 x 1	₹ 837

THE KEY ATTRIBUTES OF UNIVERSAL SHOULDER IMMOBILIZER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - A Universal Shoulder Immobilizer is a medical device designed to restrict movement of the shoulder and arm, providing support and stability during recovery from injuries or surgeries.
 - It has adjustable straps, ability to fit either the left or right arm, and a design that minimizes movement while allowing for a comfortable fit.
 - Designed to accommodate a wide range of arm and shoulder sizes.
 - It can be worn on either the left or right arm.
 - Adjustable straps allow for a secure and personalized fit and provide the necessary support to immobilize the shoulder and arm.
- **Uses/Instructions:**
 - Supports dislocated shoulder. Used for immobilisation in case of fractures of Clavicle bone and shoulder joint.
 - A Universal Shoulder Immobilizer is a device designed to restrict shoulder movement for various reasons, from injuries to post-surgical recovery. It features adjustable straps, padding and a pouch to support the arm.
 - The immobilizer keeps the arm close to the body and helps with recovery by providing support and stability.

Wrist Wrap with Thumb



Product Name	Pack Size	MRP
Wrist Wrap with Thumb	1x1	₹ 375

THE KEY ATTRIBUTES OF WRIST WRAP WITH THUMB ARE AS FOLLOWS:

- **Characteristics/Features:**
 - It is a device used to support and stabilize the wrist and thumb, often used for injury recovery, pain relief, or to enhance performance during activities like weightlifting.
 - It comprises of a thumb loop for secure placement, compression and support to reduce pain and swelling with adjustable straps for a customizable fit.
 - **Adjustable Straps:** Straps, allow for a customized fit and level of compression, ensuring comfort and security.
 - Made from breathable materials like neoprene, spandex or cotton, providing flexibility and moisture-wicking properties.
 - Designed to allow for a full range of motion in the thumb and fingers while providing support to the wrist.
- **Uses/Instructions:**
 - The wrist wrap with thumb provides compression to the wrist joint, reducing swelling and pain while also supporting the joint.
 - Used to stabilize the wrist and thumb joint, providing compression and relief from pain and inflammation.
 - Offers compression and support to the wrist and hand.
 - Used for conditions like sprains, strains, arthritis or post-operative recovery.





Product Name	Pack Size	MRP
Ankle Support	1x1	₹ 568

THE KEY ATTRIBUTES OF ANKLE SUPPORT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Ankle supports or braces, offer various levels of support and stability for the ankle joint, ranging from simple compression sleeves to more rigid, hinged braces.
 - Made from thick knitted fabric that retains body heat and provides therapeutic warmth offering a customizable fit and are good for moderate to severe sprains and sports-related injuries.
- **Uses/Instructions:**
 - Used to immobilize the joint while providing heat and compression to the bones.
 - Useful in injury rehabilitation processes that affect the ankle, being made of rigid fabric such as nylon and neoprene that allow limited mobility of the foot.
 - Helps with mild to severe sprains, instability and even post-operative recovery.
 - Offers more support with adjustable straps and other mechanisms to help stabilize the ankle joint in moderate sprains and chronic instability.



Product Name	Pack Size	MRP
Hand Stick	1x1	₹ 980


THE KEY ATTRIBUTES OF HAND STICK SUPPORT ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Offers various features to aid in mobility. Single-legged sticks provide a basic level of support, while quadripods offer greater stability due to their wider base.
 - Has an adjustable height, ergonomic handles and non-slip tips.
 - Designed for a comfortable and secure grip, reducing hand fatigue.
- **Uses/Instructions:**
 - Supports, both single-tip and four-leg (quadripod), are primarily used to aid in walking and provide stability, especially for individuals with balance issues, injuries or mobility limitations.
 - The four-leg version offers a wider base of support, which can be beneficial for those needing greater stability.
 - Quadripod sticks provide a wider base of support, making them ideal for individuals who need extra stability due to balance issues, weakness or recovering from injuries.
 - Help maintain balance and reduce the risk of falls.
 - They assist in walking, particularly for those with leg weakness or pain.



Parenteral Drugs



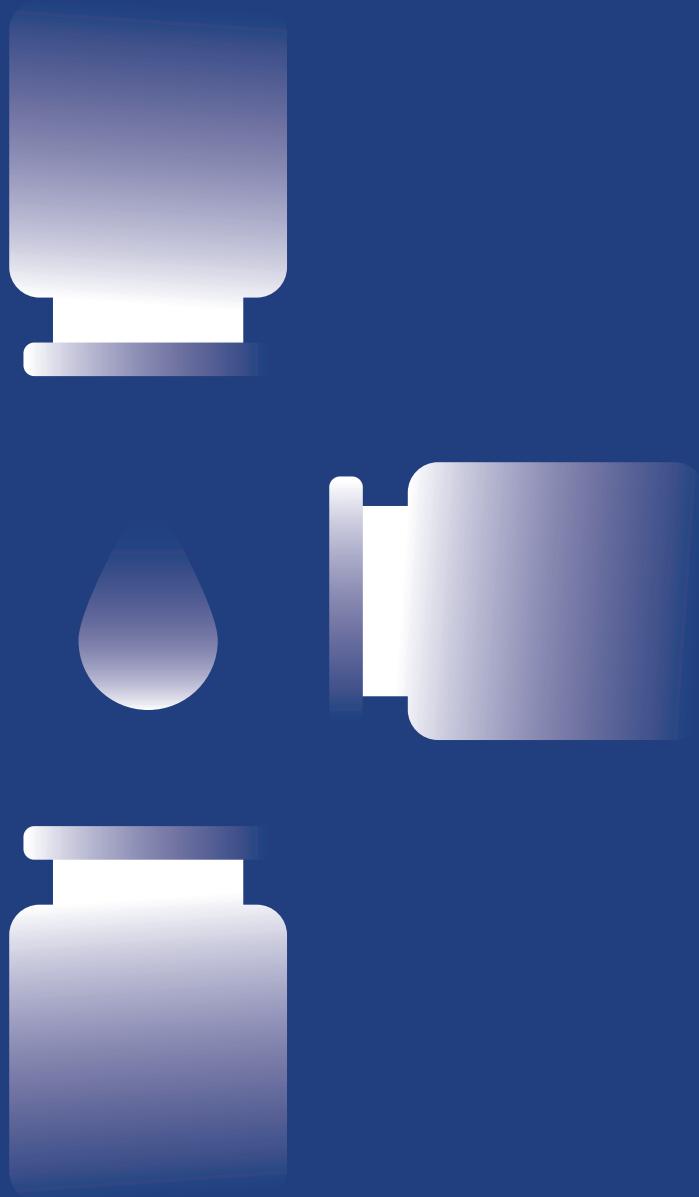



MRP: Rs. 4.00 / PC

JANITROPINE

Ingredient : Atropine Sulphate 0.6 mg + Chlorobutol 0.5 w/v

Dosage: Injection

Packaging: 1ml / Ampoule

Product Information : Atropine Sulphate Injection is an anticholinergic medication. It works by blocking the activity of a chemical messenger (acetylcholine). This helps dry up secretion (saliva, sweat, etc) from glands decreases intestinal contraction (spasms). It also works as an antidote in certain types of poisoning and reverses the side effects of certain muscle relaxing medicines.



MRP: Rs. 62.82

JANIKACIN 250 mg

Ingredient : Amikacin Sulphate 250 mg + Methyl Paraben 0.08% w/v + Propyl Paraben 0.02% w/v

Dosage: Injection

Packaging: 2ml / Vital

Product Information: Amikacin is an antibiotic. It is used in the treatment of bacterial infection. It stops bacterial growth by preventing synthesis of essential proteins required by bacteria to grow.



MRP: Rs. 113.7

JANIKACIN 500 mg

Ingredient : Amikacin Sulphate 500 mg + Methyl Paraben 0.08% w/v + Propyl Paraben 0.02% w/v

Dosage: Injection

Packaging: 2ml / Vital

Product Information: Amikacin is an antibiotic. It is used in the treatment of bacterial infection. It stops bacterial growth by preventing synthesis of essential proteins required by bacteria to grow.



MRP: Rs. 134.00



MRP: Rs. 50.00



MRP: Rs. 6.02

JANICETAM

Ingredient: Piracetam 200 mg

Dosage: I.V. Injection

Packaging: 15ML / Vial

Product Information : Piracetam Injection is used in the treatment of Alzheimer's disease and memory loss in Parkinson's disease. It can also be used to treat age-related memory loss and head injury. It works by improving communication between the nerve cells and protects the brain.

JANTHAMIN

Ingredient : Thiamine Hydrochloride 100 mg + Benzyl Alcohol 2% w/v

Dosage: I.M. / I.V. Injection

Packaging: 2ml / Ampoule

Product Information : Thiamine Hydrochloride Injection is a supplement of vitamin B1. It is used in the treatment of vitamin B1 deficiency. It is an important nutrient which helps the body to perform various functions. It can also be helpful in digestive problems including poor appetite, ulcerative colitis and ongoing diarrhoea.

JANISTIDINE

Ingredient: Ranitidine Hydrochloride 25 mg + Phenol 0.5% w/v

Dosage: I.M / I.V. Injection

Packaging: 2ml / Ampoule

Product Information : Ranitidine Hydrochloride Injection is a medicine that reduces the amount of acid stomach makes. It is used to treat and prevent heartburn, indigestion and other symptoms caused by too much acid in the stomach. It is also used to treat and prevent stomach ulcers, reflux disease and other hyperacidity conditions.



MRP: Rs. 20.75

JANIFENAC

Ingredient : Diclofenac Sodium 75 mg + Benzyl Alcohol 4.0% w/v

Dosage: I.M. / I.V. Injection

Packaging: 1ml / Ampoule

Product Information : Diclofenac Sodium belongs to a group of medicines called non steroidal anti-inflammatory drugs (NSAIDs). It is used for short-term relief of pain, inflammation and swelling in conditions that affect joints and muscles. It works by blocking chemical messengers in the brain that tell us we have pain. It can help relieve pain in conditions like rheumatoid arthritis and osteoarthritis.



MRP: Rs. 10.47


JANTAMICIN

Ingredient : Gentamicin Sulphate 40m + Methyl Paraben 0.18% w/v + Propyl Paraben 0.02% w/v

Dosage: I.M. / I.V. Injection

Packaging: 2ml / Ampoule

Product Information : Gentamicin Injection is an antibiotic used to prevent or treat a wide variety of bacterial infections. This may include infections of that urinary tract, bones and joints, lungs (e.g. pneumonia), brain (e.g. meningitis), sepsis and some others. However, it is not effective for gonorrhoea.

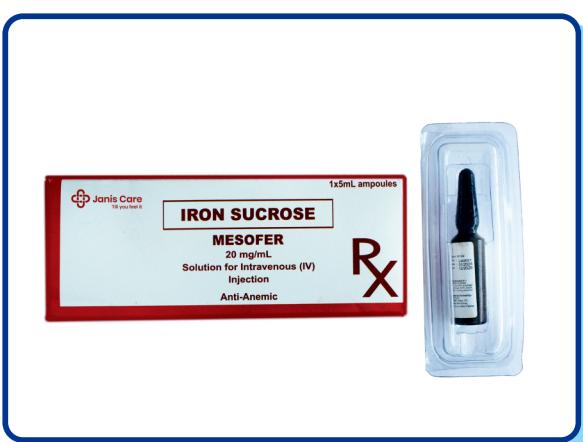

MRP: Rs. 335.00

JANIFER

Ingredient : Ferric Hydroxide 20 mg

Dosage: I.V. Injection

Packaging: 5ml / Ampoule

Product Information : Iron sucrose injection is in a class of medications called iron replacement products. It works by replenishing iron stores so that the body can make more red blood cells. Intravenous iron sucrose injection is effective in significantly raising haemoglobin levels in anaemic individuals. It is used to treat iron deficiency anaemia (not enough iron in the blood) in patients with chronic kidney disease (CKD).





MRP: Rs. 118.78



MRP: Rs. 335.77



MRP: Rs. 56.6

JANISPARIN

Ingredient : Heparin Sodium 5000 IU + Benzyl Alcohol 0.95% v/v.

Dosage: I.V. / Subcutaneous Injection

Packaging: 5ml / Vial

Product Information : Heparin Injection is an anticoagulant which prevent abnormal clotting of blood. It not only prevents the formation of harmful blood clots in the legs, lungs and heart but also stops them from growing bigger.

JANISPARIN

Ingredient : Heparin Sodium 25000 IU + Benzyl Alcohol 0.955 v/v.

Dosage: I.V. / Subcutaneous Injection

Packaging: 5ml / Vial

Product Information : Heparin Injection is an anticoagulant which prevents abnormal clotting of blood. It not only prevents the formulation of harmful blood clots in the legs, lungs and heart but also stops them from growing bigger.

JANISPAN

Ingredient: Pantoprazole Sodium 40 mg

Dosage: I.V. Injection

Packaging: 10ml / Vial

Product Information : Pantoprazole injection is used to treat hyperacidity (a certain conditions in which there is too much acid in the stomach.) It is used to treat gastroesophageal reflux disease (GERD). It is used as a short-term treatment to treat gastroesophageal reflux disease (GERD; a condition in which backward flow of acid from the stomach causes heartburn and possible injury of the esophagus [the tube between the throat and stomach]) in people who have had damage to their esophagus.

Parenteral Drugs - Injections



MRP: Rs. 1067.00



MRP: Rs. 809.74



MRP: Rs. 69.86

JANISPENEM

Ingredient : Meropenem Trihydrate 1000mg + Sodium Carbonate 90.2mg

Dosage: I.V. Injection

Packaging: 20ml / Vial

Product Information: This Injection is an antibiotic that is used to treat severe infections of the skin, lungs, stomach, urinary tract, blood and brain (e.g. meningitis). It works by killing the bacteria that cause these problems. However, it will not treat a viral infection. This is commonly used to treat critically ill patients admitted to the hospital. This medicine is given by drip/infusion or by direct injection into a vein, under the supervision of a doctor or a nurse.

JANISPENEM

Ingredient : Meropenem Trihydrate 500 mg + Sodium Carbonate 45.1mg

Dosage: I.M. / I.V. Injection

Packaging: 10ml / Vial

Product Information: This Injection is an antibiotic that is used to treat severe infections of the skin, lungs, stomach, urinary tract, blood and brain (e.g. meningitis). It works by killing the bacteria that cause these problems. However, it will not treat a viral infection. This is commonly used to treat critically ill patients admitted to the hospital. This medicine is given by drip/infusion or by direct injection into a vein, under the supervision of a doctor or a nurse.

JANTRIAZONE

Ingredient : Ceftriaxone Sodium 1000 mg

Dosage: I.M. / I.V. Injection

Packaging: 10ml / Vial

Product Information: This injection is an antibiotic belonging to the cephalosporin group, which is used to treat bacterial infections in your body. It is effective in infection of the brain (e.g. meningitis), lungs (e.g. pneumonia), ear, urinary tract, skin & soft tissues, bones & joints, blood and heart.



MRP: Rs. 562.31



MRP: Rs. 453.69



MRP: Rs. 13.35

JANICYCLINE

Ingredient : Doxycycline Hydrochloride 100 mg

Dosage: I.V. Injection

Packaging: 10ml / Vial

Product Information : Doxycycline Injection is an antibiotic medicine used to treat or prevent bacterial infections in body. It is effective in some infections of the lungs, urinary tract, eyes and others. It kills bacteria which helps to improve symptoms and cure the infection. It is also used to treat many sexually transmitted diseases. It helps to improve symptoms and cure the underlying infection. It is given as a drip (intravenous infusion).

JANITAZ

Ingredient : Piperacillin Sodium 4 gm + Tazobactam Sodium 0.5 gm

Dosage: I.V. Injection

Packaging: 20ml / Vial

Product Information : Piperacillin and tazobactam combination is an antibiotic that belongs to the group of medicines known as penicillins and beta-lactamase. It works by killing the bacteria and preventing their growth. It is used to treat bacterial infections in many different parts of the body (eg, stomach or bowel, lungs, skin, female reproductive organs).

JANITRONE

Ingredient: Ondansetron Hydrochloride 4 mg

Dosage: I.V. / I.M. Injection

Packaging: 2 ml / Ampoule

Product Information : This Injection is an antiemetic commonly used to control nausea and vomiting due to certain medical conditions like stomach upset. It is also used to prevent nausea and vomiting caused due to any surgery, cancer drug therapy or radiotherapy.



MRP: Rs. 42.50

J D5 - L - 500 ML

Composition: Dextrose 5.0% w/v

Dosage: I.V. Fluids

Packaging: 500 ml Bottle

Major Indications: Provides free water to help renal excretion of solutes, hypernatremia and some dextrose supplementation.



MRP: Rs. 23.50

JANIS METZOLE IV 500 MG

Composition: Metronidazole 500 mg + Sodium Chloride 0.8%

Dosage: I.V. Fluids

Packaging: 100 ml Bottle

Major Indications : It is used to treat serious bacterial infections in different areas of the body. It is also used to prevent infections in the bowels before and after surgery for some patients. Metronidazole belongs to the class of medicines known as antibiotics. It works by killing bacteria or preventing their growth.



MRP: Rs. 623.00

JANISPAR IV 100 ML

Composition: Paracetamol IP 1000 mg

Dosage: I.V. Fluids

Packaging: 100 ml Bottle

Major Indications: Janispar IV is a medicine used to treat mild to moderate pain. It can also be used to treat fever (high temperature). It's dangerous to take more than the recommended dose of paracetamol. Janispar overdose can damage liver and cause death.



100 ml -MRP: Rs. 22.00

500 ml -MRP: Rs. 39.00

1000 ml -MRP: Rs. 65.25

J NS-L-100 ML / 500 ML / 1000 ML

Composition: 0.9% Sodium Chloride w/v

Dosage: I.V. Fluids

Packaging: 100 ml / 500 ml / 1000 ml Bottle

Major Indications : Fluid resuscitation for haemorrhaging, severe vomiting, diarrhoea, GI suctioning losses, wound drainage, mild hyponatraemia or blood transfusions.



MRP: Rs. 42.79

J DNS-L-500 ML

Composition: Sodium Chloride 0.9% w/v & Dextrose 5.0% w/v

Dosage: I.V. Fluids

Packaging: 500 ml Bottle

Major Indications : It is a combination of two medicines Dextrose and Sodium Chloride. They work by restoring blood plasma lost due to severe bleeding. Plasma is required to circulate red blood cells that deliver oxygen throughout the body.



MRP: Rs. 63.28

JANISRIN L 500 ML

Composition: Lactic Acid 0.24 ml + Sodium Hydroxide 0.115 gm + Sodium Chloride 0.6 gm + Potassium Chloride 0.04 gm + Calcium Chloride 0.027 gm

Dosage: I.V. Fluids


Packaging: 500 ml Bottle

Major Indications : Fluid resuscitation, GI tract fluid losses, burns, traumas or metabolic acidosis. Often used during surgery.





Baby Care

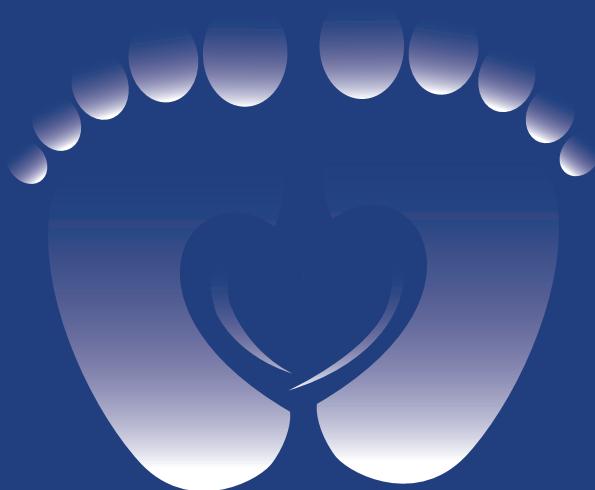



BABY DIAPERS – Pack Of 10

Sizes	Pack Size	MRP
Baby Diapers - M	1 x 10	₹ 1150
Baby Diapers - L	1 x 10	₹ 1360
Baby Diapers - XL	1 x 10	₹ 1565

BABY DIAPERS – Pack Of 5

Sizes	Pack Size	MRP
Baby Diapers - M	1 x 5	₹ 800
Baby Diapers - L	1 x 5	₹ 850
Baby Diapers - XL	1 x 5	₹ 900
Baby Diapers - XXL	1 x 5	₹ 950

THE KEY ATTRIBUTES OF BABY DIAPERS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Baby diapers are designed to absorb and contain baby waste, keeping them dry and comfortable. Made from skin friendly, highly absorbent materials. They are highly absorbent, breathable and soft as well as leakproof and a snug fit design. Made from soft, non-irritating materials.
 - Inner layer is made up of polymer based material that protects baby's skin.
 - Outer layer is made of polyethylene to prevent leakage.
 - Top Sheet: Allows urine to flow through keeping liquid away from baby's skin and retains absorbent core structure.
 - Easy-to-use closures and a lightweight design make diaper changes simpler.
- **Uses/Instructions:**
 - Used to keep babies dry and comfortable, preventing leaks and protecting their clothing and bedding.
 - The absorbent material in diapers can be used to absorb spills or leaks.
 - Diapers help maintain hygiene by keeping skin dry and preventing leaks.
 - Diapers reduce the need for frequent laundry and cleaning, saving time and effort.
 - Diapers allow babies to move freely without the discomfort of a wet or soiled diaper.



BABY DIAPERS - PANTS / PULL-UPS - Pack Of 10

Sizes	Pack Size	MRP
Baby Diapers - M	1 x 10	₹ 1050
Baby Diapers - L	1 x 10	₹ 1160
Baby Diapers - XL	1 x 10	₹ 1265

BABY DIAPERS - PANTS / PULL-UPS - Pack Of 5

Sizes	Pack Size	MRP
Baby Diapers - M	1 x 5	₹ 550
Baby Diapers - L	1 x 5	₹ 600
Baby Diapers - XL	1 x 5	₹ 650
Baby Diapers - XXL	1 x 5	₹ 690

THE KEY ATTRIBUTES OF BABY PANTS ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Designed for Quick and easy changes. Perfect for standing or active babies who may not stay still during diaper changes. Are a part of potty training.
 - The flexible waistband assists in potty training process by making it easier for the child to pull the diaper up and down by themselves.
 - They are equipped with a highly absorbent core, kids' pull-ups can hold substantial amounts of leakage, keeping the child dry and comfortable for extended periods.
- **Uses/ Instructions**
 - When the baby is 6 months old, they often flip over onto their stomachs, rolling and crawling. They do not stay still. This means that it becomes difficult to put them on their backs to put on tape diapers. So, during this age, putting on and removing pant diapers is easier and more convenient.
 - They are designed for a secure and comfortable fit, 360° stretchy waistband, leak protection and no tape adjustments—making them ideal for active babies and busy parents.
 - Unlike regular nappies that you wrap around your baby's bottom and secure with tapes at the side, nappy pants have an all-round waistband that makes them ideal for active babies and toddlers who are less willing to lie still for a nappy change.



Product Name	Pack Size	MRP
Baby Wipes	1 x 72	₹ 603

THE KEY ATTRIBUTES OF BABY WIPES ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Our Baby Wipes are approved by Skin Health Alliance and accepted by The National Eczema Association.
 - Baby wipes are pre-moistened wipes designed for cleaning babies' skin, especially during diaper changes
 - Made from soft, non-woven fabric, gentle cleaning solutions and offer a convenient packaging.
- **Uses/Instructions:**
 - Baby wipes are primarily used for cleaning a baby's skin during diaper changes and quick cleanups.
 - Also used for disinfecting surfaces and cleaning various items.



Medicines







MRP: Rs. 19.00

PAIGO 500 MG

Ingredient : Paracetamol 500 mg

Dosage: Tablets

Packaging: 1*15

Product Information : Paigo 500 mg Tablet helps to relieve pain and fever blocking the release of certain chemical messengers responsible for fever and pain. It is used to treat headache, migraine, toothache, sore throat, menstrual pain (Dysmenorrhea), arthritis, muscle aches and common cold.



MRP: Rs. 32.00

PAIGO 650 MG

Ingredient: Paracetamol 650 mg

Dosage: Tablets

Packaging: 1*15

Product Information : Paigo 650 mg Tablet helps to relieve pain and fever blocking the release of certain chemical messengers responsible for fever and pain. It is used to treat headache, migraine, toothache, sore throat, menstrual pain (Dysmenorrhea), arthritis, muscle aches and common cold.



MRP: Rs. 95.00

PAIGO PLUS

Ingredient : Aceclofenac 100 mg+ Paracetamol 325 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Paigo Plus works as pain reliever and fever reducer. It is used to treat Rheumatoid arthritis, Osteoarthritis, Ankylosing spondylitis, Back pain, Sprain, Strain, Sports injuries, Dental pain, Dysmenorrhea etc.





MRP: Rs. 139.00

PAIGO PLUS SP

Ingredient : Aceclofenac 100 mg + Paracetamol 325 mg + Serratiopeptidase 15 mg

Dosage: Tablets

Packaging: 1*10

Product Information: Paigo Plus SP is used in the treatment of rheumatoid arthritis, osteoarthritis and ankylosing spondylitis. It is also useful in acute and chronic pain, inflammatory conditions, post operative pains, dental conditions, traumatic injuries and oedema.



MRP: Rs. 208.00

PAPOL D

Ingredient: Pantoprazole 40 mg + Domperidone 30 mg

Dosage: Capsules

Packaging: 1*10

Product Information : Papol D Capsule is a prescription medicine used to treat gastroesophageal reflux disease (Acid reflux), dyspepsia (indigestion) and gastritis. It helps treat the conditions by reducing the amount of acid in the stomach thereby relieving symptoms of acidity such as heartburn, stomach pain or irritation.



*GST will be applicable on the rates of all the products.



MRP: Rs. 165.00



MRP: Rs. 375.00



MRP: Rs. 515.00

JUMPON - L 50 MG

Ingredient: Sildenafil Citrate (Oxide Blue) 50 mg

Dosage: Tablets

Packaging: 1*4

Product Information : Jumpon - L 50 mg Tablet is a prescription medicine used to treat erectile dysfunction (impotence) in men. It works by increasing blood flow to the penis. This helps men to get and maintain hard erection.

JUMPON - S 100 MG

Ingredient: Sildenafil Citrate (Oxide Blue) 50 mg

Dosage: Tablets

Packaging: 1*4

Product Information : Jumpon - S 100 mg Tablet is a prescription medicine used to treat erectile dysfunction (impotence) in men. It works by increasing blood flow to the penis. This helps men to get and maintain hard erection.

ERECTROD 20 MG

Ingredient: Tadalafil 20 mg

Dosage: Tablets

Packaging: 1*4

Product Information : Erectrod 20 mg Tablet is used to treat erectile dysfunction (ED) in men and pulmonary arterial hypertension (PAH). It increases blood flow to the penis to help men get and maintain hard erection. In PAH, it relaxes or widens the blood vessels inside the lungs, which reduces blood pressure in the lungs.

ETHICAL MEDICINES Calcium Supplement



MRP: Rs. 80.10



MRP: Rs. 149.00

ETHICAL MEDICINES Anti Reflux



MRP: Rs. 135.00

JANICAL - D3

Ingredient: Calcium & Vitamin D3

Dosage: Tablets

Packaging: 1*15

Product Information : Janical-D3 is a dietary supplement which is primarily useful for managing Vitamin D and Calcium deficiencies. It contains Calcium and Vitamin D3 as its main ingredients. Calcium an essential mineral that plays a vital role in maintaining bone formation and maintenance. The presence of Vitamin D3 in it improves the absorption of calcium and enhance bone health. It is also useful for bone related conditions such as joint pains, immunity issues, low energy and lethargy.

JANISTITE SYRUP

Ingredient: Cyproheptadine hydrochloride 2 mg + Tricholine Citrate 0.275 gm + Sorbitol Solution 2 gm

Dosage: Syrup

Packaging: 1*1

Product Information : Janistite Syrup is a combination medicine used to treat loss of appetite. It is an effective appetite stimulant. It works by reducing the effect of a chemical messenger which regulates appetite.

JANISEC - D

Ingredient: Pantoprazole 40 mg + Domperidone 10 mg

Dosage: Capsule

Packaging: 1*10

Product Information : Janisec - D Capsule is a prescription medicine used to treat gastroesophageal reflux disease (Acid reflux), dyspepsia (indigestion) and gastritis. It helps treat the conditions by reducing the amount of acid in the stomach thereby relieving symptoms of acidity such as heartburn, stomach pain or irritation.



***GST will be applicable on the rates of all the products.**



MRP: Rs. 17.00

JANIDOL 500 MG

Ingredient: Paracetamol 500 mg

Dosage: Tablets

Packaging: 1*15

Product Information: Janidol 500 mg Tablet helps to relieve pain and fever by blocking the release of certain chemical messengers responsible for fever and pain. It is used to treat headache, migraine, toothache, sore throat, menstrual pain (Dysmenorrhea), arthritis, muscle aches and common cold.



MRP: Rs. 30.00

JANIDOL 650 MG

Ingredient: Paracetamol 650 mg

Dosage: Tablets

Packaging: 1*15

Product Information: Janidol 650 mg Tablet helps to relieve pain and fever by blocking the release of certain chemical messengers responsible for fever and pain. It is used to treat headache, migraine, toothache, sore throat, menstrual pain (Dysmenorrhea), arthritis, muscle aches and common cold.



MRP: Rs. 85.00

JANIDOL PLUS

Ingredient: Aceclofenac 100 mg + Paracetamol 325 mg

Dosage: Tablets

Packaging: 1*10

Product Information: Janidol Plus works as pain reliever and fever reducer. It is used to treat rheumatoid arthritis, Osteoarthritis, Ankylosing spondylitis, Back pain, Sprain, Strain, Sports injuries, Dental pain, Dysmenorrhea etc.



MRP: Rs. 130.00

JANIDOL PLUS SP

Ingredient : Aceclofenac 100 mg + Paracetamol 325 mg + Serratiopeptidase 15 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janidol Plus SP is used in the treatment of rheumatoid arthritis, osteoarthritis and ankylosing spondylitis. It is also useful in acute and chronic pain, inflammatory conditions, post operative pains, dental conditions, traumatic injuries and oedema.



MRP: Rs. 55.00

JANIFENAC - P

Ingredient: Diclofenac 50 mg + Paracetamol 325 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janifenac - P Tablet for pain relief. It relieves pain in conditions like headache, migraine, muscle pain, dental pain, rheumatoid arthritis, ankylosing spondylitis, osteoarthritis and Dysmenorrhea.



MRP: Rs. 21.00

JANISPAS

Ingredient: Mefenamic Acid Tablet 500 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janispas is used in the treatment of various painful conditions like menstrual pain, headache, muscle pain, joint pain and dental pain. It effectively alleviates pain and inflammation.



MRP: Rs. 240.00

JANIMONT - L

Ingredient: Levocetirizine Hcl 5 mg + Montelukast Sodium

10 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janimont - L Tablet is a combination medicine used in the treatment of allergic symptoms such as runny nose, stuffy nose, sneezing, itching, swelling, watery eyes and congestion or stuffiness. It also reduces inflammation in the airways and makes breathing easier.



MRP: Rs. 18.50

JANICET

Ingredient: Cetirizine Dihydrochloride 10 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janicet Tablet belongs to a group of medicines called antihistamines. It used to treat various allergic manifestations such as hay fever, conjunctivitis, some skin reactions and reaction to bites and stings. It relieves watery eyes, runny nose, sneezing and itching.



MRP: Rs. 58.00

JANISPAS - D

Ingredient : Dicyclomine Hydrochloride 10 mg + Mefenamic Acid 250 mg

Dosage: Tablets

Packaging: 1*10

Product Information : Janispas - D Tablet is a prescription medicine that helps to provide symptomatic relief from menstrual pain and cramps. It is also used to treat abdominal pain by relieving spasms of the muscles in the stomach and intestines.


MRP: Rs. 151.00

30 gm MRP: Rs. 82.00
20 gm MRP: Rs. 63.00

15 gm MRP: Rs. 65.00
30 gm MRP: Rs. 95.00

JANICLO DUSTING POWDER

Ingredient: Clotrimazole (1% w/w)

Dosage: Powder

Packaging: 100 gm

Major Indications : Janiclo Dusting Powder is primarily useful for fungal infection of the skin such as athlete's foot, jock itch, ringworm and other fungal skin infections. This medication is also beneficial for the skin conditions known as pityriasis (Flaking or scaling of the skin).

JANICLO 1% CREAM

Ingredient in every injection product: Clotrimazole (1% w/w)

Dosage: Cream

Packaging: 30 gm / 20 gm

Major Indications : Janiclo 1% Cream is primarily useful for fungal infections of the skin such as athlete's foot, jock itch, ringworm and other fungal skin infections. This medication is also beneficial for the skin conditions known as pityriasis (Flaking or scaling of the skin).

JANICLO - B CREAM

Ingredient: Beclomethasone (0.025 % w/w) + Clotrimazole (1% w/w)

Dosage: Cream

Packaging: 15 gm / 30 gm

Major Indications : Janiclo - B Cream is a combination medicine. It is used to treat various types of skin infections usually the fungal type of skin infections. It minimizes symptoms of inflammation such as redness, swelling and itching. It also acts against the infection causing micro organisms.



MRP: Rs. 395.00

JANIREC 20 MG

Ingredient: Tadalafil 20 mg

Dosage: Tablets

Packaging: 1*4

Major Indications : Janirec 20 mg Tablet is used to treat erectile dysfunction (ED) in men and pulmonary arterial hypertension (PAH). It increases blood flow to the penis to help men get hard erection. In PAH it relaxes or widens the blood vessels inside the lungs, which reduces blood pressure in the lungs.



MRP: Rs. 160.00

JANION - L 50 MG

Ingredient: Sildenafil Citrate (Oxide Blue) 50 mg

Dosage: Tablets

Packaging: 1*4

Major Indications : Janion - L 50 mg Tablet is a prescription medicine used to treat erectile dysfunction (impotence) in men. It works by increasing blood flow to the penis. This helps men to get and maintain hard erection.



MRP: Rs. 365.00

JANION - S 100 MG

Ingredient: Sildenafil Citrate (Oxide Blue) 100 mg

Dosage: Tablets

Packaging: 1*4

Major Indications : Janion - S 100 mg Tablet is a prescription medicine used to treat erectile dysfunction (impotence) in men. It works by increasing blood flow to the penis. This helps men to get and maintain hard erection.



MRP: Rs. 79.00



MRP: Rs. 85.00



MRP: Rs. 114.00

JANITHRO 250 MG

Ingredient: Azithromycin 250 mg

Dosage: Tablets

Packaging: 1*6

Major Indications : Janithro 250 mg Tablet is an antibiotic used to treat various types of bacterial infections of the respiratory tract, ear, nose, throat, lungs, skin and eye in adults and children. It is also effective in typhoid fever and some sexually transmitted disease like gonorrhoea.

JANITHRO 500 MG

Ingredient : Azithromycin 500 mg

Dosage: Tablets

Packaging: 1*3

Major Indications : Janithro 250 mg Tablet is an antibiotic used to treat various types of bacterial infections of the respiratory tract, ear, nose, throat, lungs, skin and eye in adults and children. It is also effective in typhoid fever and some sexually transmitted disease like gonorrhoea.


JANIFLOXACIN

Ingredient : Ofloxacin 200 mg and Ornindazole 500 mg


Dosage: Tablets

Packaging: 1*10

Major Indications : Janifloxacin Tablet is a combination of two antibiotics. It is used in the treatment of bacterial and parasitic infection. It effectively treats a wide range of bacterial infections that may occur in the teeth, lungs, gastrointestinal tract, urinary and genital tract.

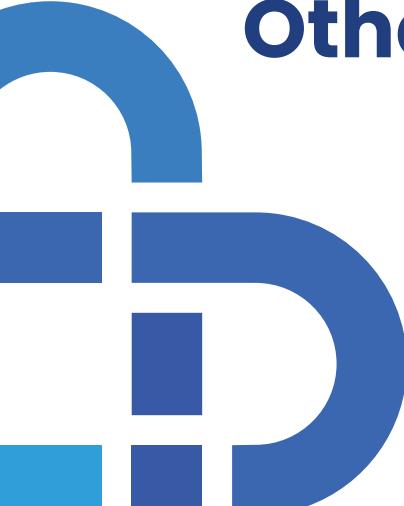

Others

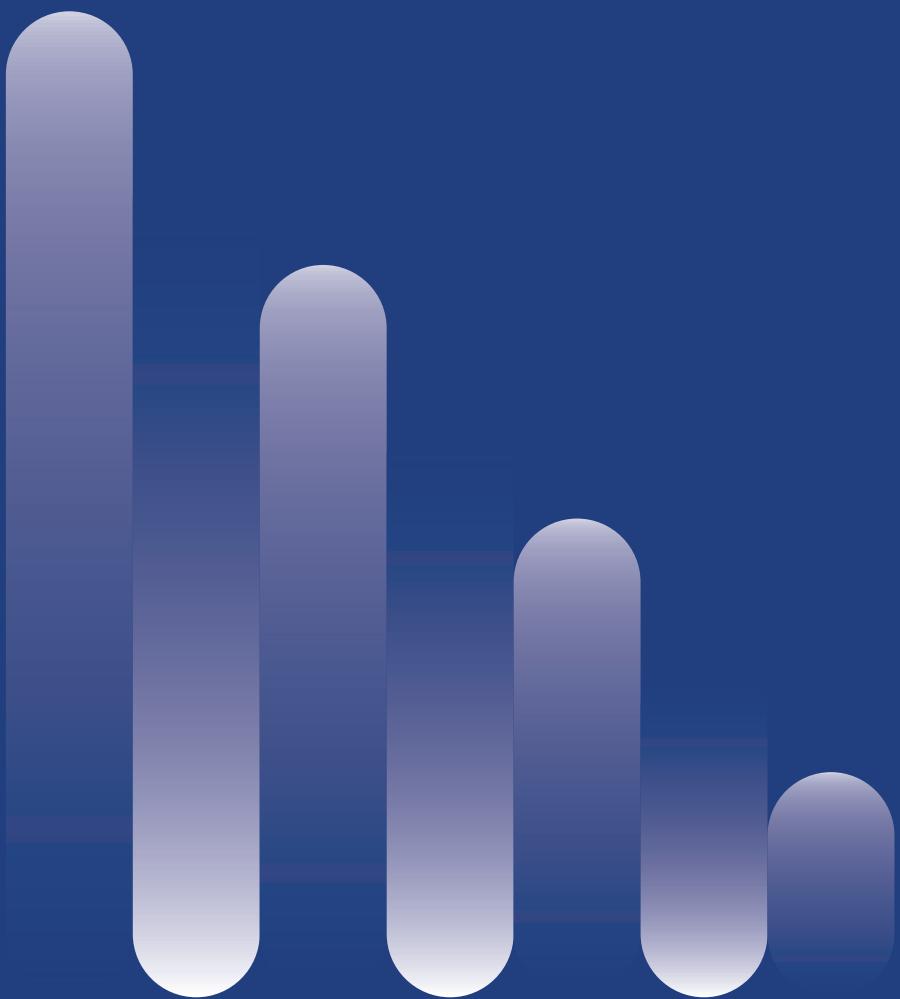



Product Name	Pack Size	MRP
Dialyzer (14)	1x1	₹ 1400
Dialyzer (16)	1x1	₹ 1400

THE KEY ATTRIBUTES OF DIALYZER ARE AS FOLLOWS:

- **Characteristics/Features:**
 - A dialyzer, often called an "artificial kidney," is a vital component of hemodialysis, separating blood and dialysate to remove waste and excess fluid. Key characteristics include a semipermeable membrane, four ports for blood, dialysate, varying membrane materials and surface areas.
 - Incorporates a dialysis membrane which is a thin film that separates blood and dialysate, allowing small molecules (waste and excess fluids) to pass through while retaining blood cells and proteins.
 - Has four ports: one inlet and one outlet for both blood and dialysate.
 - Inside the dialyzer, the blood flows through a semi-permeable membrane, while a dialysis fluid (dialysate) flows in the opposite direction.
- **Uses/Instructions:**
 - A dialyzer, often called an "artificial kidney," is a device used in dialysis to filter and purify blood when a patient's kidneys are unable to perform this function adequately. It removes excess waste and fluid from the blood, allowing the clean blood to return to the body.
 - Blood is drawn from the patient, typically from a vein in the arm and pumped into the dialyzer.
 - The cleaned blood, now free of excess waste and fluid, is then returned to the patient's circulation through the second needle.
 - Suited for diffusive clearance, commonly used in intermittent hemodialysis.



Product Name	Pack Size	MRP
AV Fistula Needle (16)	1x1	₹ 160
AV Fistula Needle (17)	1x1	₹ 160

THE KEY ATTRIBUTES OF AV FISTULA NEEDLE ARE AS FOLLOWS:

- **Characteristics/Features:**
 - Specially designed for dialysis, offering enhanced safety, comfort and efficacy compared to standard needles.
 - Finely polished needles facilitate the puncture easily.
 - Provided with flexible butterfly wings for proper fixation.
 - Back eye and ultra thin walls ensure high blood flow rate.
- **Uses/Instructions:**
 - Used to connect blood lines to the blood vessel through the needle when the dialysis is carried out via an internal fistula.



Product Name	Pack Size	MRP
Sodium Hypochlorite	1x1	₹ 300

THE KEY ATTRIBUTES OF SODIUM HYPOCHLORITE ARE AS FOLLOWS:

- **Characteristics/Features:**
- Sodium hypochlorite (NaClO), commonly known as bleach, is a pale greenish-yellow liquid with a strong chlorine odor.
- **Uses/Instructions:**
- It is a strong oxidizer and a weak base, making it effective as a disinfectant, bleaching agent and deodorizer. Sodium hypochlorite is unstable, especially in its free form, so it's typically used as an aqueous solution.
- Used to clean infected topical wounds.
- Used for bleaching, disinfection and water treatment.
- It's a strong oxidizing agent, effective in killing bacteria, viruses and other microorganisms
- Also used for odour removal.



Corporate Office

Janis Care Pvt. Ltd

9th Floor, Landmark Pllatinum, S V Road,
Bandra West, Mumbai - 400050, India.

Toll Free No.: 1800 2588 039

E-mail: info@janiscare.com

Website: www.janiscare.com

Regional Office

Aurangabad Office

Janis Care Pvt. Ltd
Shop No. 01, Ground Floor, Plot No. 10
Survey No. 125/126, Shivaji Nagar, Satara,
Aurangabad, Maharashtra 431005

Delhi-Zonal Office

Janis Care Pvt. Ltd
407-408, 4th Floor, DDN Tower-1, District
Center, Janakpuri, New Delhi 110058.

Andheri Office

Janis Care Pvt. Ltd
G-14Supreme Industries, Mograpada,
opp Reliance, Near EcoSpace, Andheri
East, Mumbai 400069

Pune Office

Janis Care Pvt. Ltd
Plot No. - 139/2, Sector -23, Transport
Nagar Opp. Nitco Office, Bhakti Shali
Chowk Nigdi, Pune, Maharashtra 411044.

Jaipur Office

Janis Care Pvt. Ltd
Plot no. G1-456 (C),1st Floor, Road no. 12
Vishwakarma Industrial Area, Jaipur -
302013

Manufacturing Address

Navsari

Janis Care Pvt. Ltd
Block No. 79, Siyada Road, Khudvel,
Kaliyari, Chikhili, District, Navsari, Gujarat-
396540, India.

Toll Free No.: 1800 2588 039

(Part of Janis Group)

Life Guard Pharma Pvt. Ltd.
Plot No.169, Sector-07, IMT Manesar,
Gurugram Haryana.